

SLEEP COMFORT CARE PAD

Cleaning and Application Guide



DirectHome
MEDICAL

Presented With Compliments From
www.DirectHomeMedical.com



Comfort Care

SLEEP
COMFORT
CARE
PAD

DISTRIBUTED BY SEQUAL TECHNOLOGIES INC.

1
STEP



Clean pad with hand soap before each use.

2
STEP

Rinse pad with warm to hot tap water. The water should NOT burn your hands.



3
STEP



Blot pad dry using a towel.

4
STEP

Apply pad to your face. Place tip of pad at eyebrow level, and wings of pad 1/4" below bottom of eyes.



5
STEP



Apply your CPAP mask.

6
STEP

Ensure a secure fit.

