

## TIARA MEDICAL GAUGE MANOMETER – USER INSTRUCTIONS

If the pointer on the Pressure Manometer does not return to rest on the stop pin with no pressure applied, please see “Calibration Instructions”

### To Check Output Pressure of CPAP/BIPAP

1. Connect one end of the oxygen tubing supplied with your Manometer to the nozzle on Manometer
2. Connect the other end of oxygen tubing to the small nozzle on the oxygen connector adapter (also supplied with your Manometer)
3. Remove CPAP/BiPAP tubing from CPAP/BiPAP and insert the oxygen connector adapter in its place.
4. Turn CPAP/BiPAP pressure ON
5. To check output pressure occlude (block) the open end of the oxygen connector adapter ensuring that no air escapes.

### To Check Pressure Inline

1. Connect one end of the oxygen tubing supplied with your Manometer to the nozzle on the Manometer
2. Connect the other end of oxygen tubing to the small nozzle on the oxygen connector adapter (also supplied with your Manometer)
3. Remove CPAP/BiPAP tubing from CPAP/BiPAP and insert the oxygen connector adapter in its place.
4. Re-attach your CPAP/BiPAP tubing to the open end of the oxygen connector adapter ensuring that the seal is tight.
5. Connect your CPAP/BiPAP mask to the tubing as you normally would.
6. Place the CPAP/BiPAP mask on the way you normally would when preparing for sleep.
7. Turn CPAP/BiPAP pressure ON

