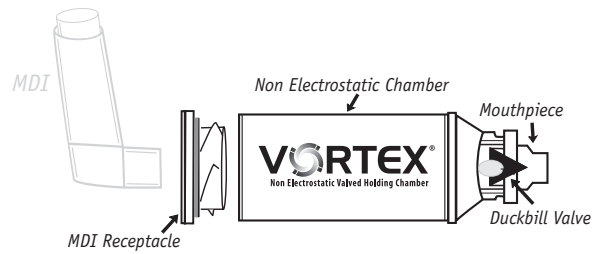
VORTEX®

Non Electrostatic Valved Holding Chamber

SINGLE PATIENT USE

LATEX FREE ■ DISHWASHER SAFE

SAVE THESE INSTRUCTIONS

CAUTION

U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Use the product only for its intended use as described in these Instructions For Use. Improper use can result in serious or fatal illness/injury.

WARNING

1. Close supervision is necessary when this product is used by children or near the physically or mentally impaired. This device contains small parts that may present a choking hazard to small children. Always keep out of the reach of children three years old and under. All parts should be seated firmly in place. Use of an improperly assembled PARI VORTEX® Non Electrostatic Valved Holding Chamber could prevent adequate delivery of medication. If not sure of the suitability and/or functioning of your MDI, consult MDI instructions and/or your physician.
2. To reduce the risk of infection, illness or injury from contamination and to reduce the risk of improper performance and treatment, you must thoroughly clean and dry all parts of the PARI VORTEX® as instructed on reverse side.
3. To reduce the risk of improper treatment, use only PARI accessories (e.g. masks).
4. To reduce the risk of improper treatment, product should be replaced if valves or parts are damaged or missing.
5. To reduce the risk of improper treatment, make sure duckbill valve is properly fitted.
6. If unsure on how to use this product, talk to your physician or pharmacist.
7. Administer medication per MDI instructions.

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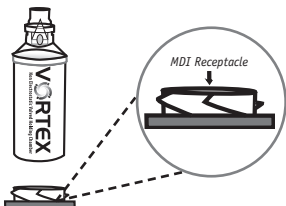
Presented With Compliments From
www.DirectHomeMedical.com

ATTACHING VORTEX® MDI RECEPTACLE / ATTACHER LA DOUILLE DE L'AÉROSOL-DOSEUR VORTEX® / CÓMO CONECTAR EL RECEPTÁCULO VORTEX® MDI

1.

Place the unit in an area that is clean and free of dust. Place the MDI Receptacle on a stable flat surface.

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2.

Align metal cylinder directly on top of MDI Receptacle and insert onto the MDI Receptacle.

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3.

Rock metal cylinder back and forth until MDI Receptacle is completely attached around the cylinder.

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4.

MDI Receptacle is now firmly attached.

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PARI VORTEX HOLDING CHAMBER
has a 6 month warranty for home use.

PARI Respiratory Equipment, Inc. | 2943 Oak Lake Blvd., Midlothian, VA 23112-3998 USA
Tel: 1.800.FAST.NEB (327.8632) and 1.804.253.7274 | Fax: 1.800.727.4112 | www.PARI.com

© 2009 PARI Respiratory Equipment, Inc. All rights reserved.
051D5000 Rev D 1-09

INSTRUCTIONS FOR USE - PARI VORTEX® Non Electrostatic Valved Holding Chamber

IMPORTANT: Follow instructions included with your metered dose inhaler (MDI)

1.

Look for foreign, damaged, and/or missing parts. If using an optional mask, firmly attach to mouthpiece.

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2.

Remove cap of metered dose inhaler (MDI) and insert into the back MDI receptacle of the PARI Vortex® as shown.

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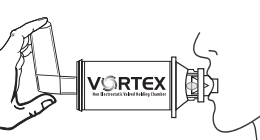
4.

Use with Mouthpiece: Place mouthpiece into mouth. Exhale, depress MDI, and then breathe in slowly and deeply. Hold breath for 10 seconds or for as long as is comfortable.

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5.

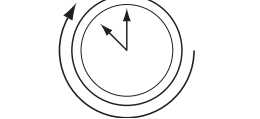
Use with Mask: Apply to face over nose and mouth creating a good seal. Depress MDI and breathe in and out slowly for 10 seconds.

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6.


Follow physician and MDI instructions before repeating steps 4-5.

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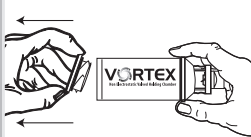Remove metered dose inhaler (MDI) receptacle. Remove mask (optional).

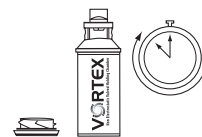
2

A. Soak parts and mask for 15 minutes in luke warm water with liquid detergent. Agitate gently.
OR
B. Place parts in top shelf of dishwasher. Hand wash mask separately.


3

A. Rinse in clean water. Shake out in sink.
OR
B. Remove from dishwasher. Shake out excess water.

4

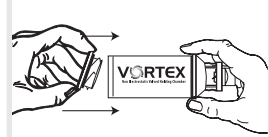Thoroughly air dry in vertical position on a clean, lint-free cloth or on a drying rack.

5

Replace MDI receptacle and mask (optional) when unit is completely dry and ready for use.

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