

# Quick Start Guide

for *patients*

866-722-2507

[www.aeiomed.com](http://www.aeiomed.com)



everest<sup>2</sup>

The everyday CPAP that can also go mobile


**Direct Home MEDICAL**  
 Presented With Compliments From  
[www.DirectHomeMedical.com](http://www.DirectHomeMedical.com)



AEIOMed, Inc.  
 1313 5th Street SE, Suite 205  
 Minneapolis, MN 55414  
[www.aeiomed.com](http://www.aeiomed.com)  
 Toll free: 866-722-2507  
 Phone: 612-455-0550  
 Fax: 612-455-0551  
 Email: [info@aeiomed.com](mailto:info@aeiomed.com)

PN101160 Rev A

Everest and AEIOMed are registered trademarks of AEIOMed, Inc. © 2008 AEIOMed, Inc. All rights reserved.




# Set-up Instructions

To begin, plug in the power connector

## Ramp Settings and Compliance Sessions

for use of CPAP with or without heated humidifier

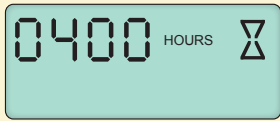


### Total Compliance Hours

Momentarily displays total hours of use before going into stand-by mode. CPAP must be in stand-by mode to adjust ramp or humidifier settings.




### Stand-by Mode

Ready to begin operation. Press button.

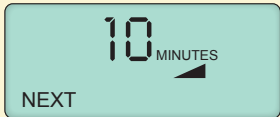

### Ramp Time Period

Time period to reach therapy pressure (set by clinician). Press NEXT to continue.

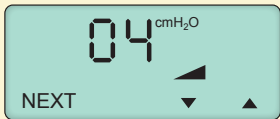


### Starting Ramp Pressure

Use or arrows to adjust starting ramp pressure. If arrows are not present, this feature has not been prescribed for you. Press NEXT to continue.



### Compliance Sessions $\geq$ 8 hours

Number of on/off sessions equal or greater than 8 hours each. Press NEXT to continue.

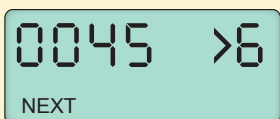


### Compliance Sessions $\geq$ 6 hours

Number of on/off sessions equal or greater than 6 hours each. Press NEXT to continue.



### Compliance Sessions $\geq$ 4 hours

Number of on/off sessions equal or greater than 4 hours each. Press NEXT to continue.

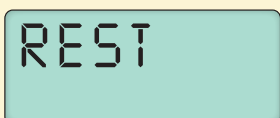

### Stand-by Mode

Ready to begin operation. To begin therapy, press . To adjust humidifier settings, see following instructions.


## Humidifier Settings

for use of CPAP with heated humidifier




### Stand-by Mode with Humidifier Reservoir in Place

Ready to begin operation with humidifier in place. Press button.

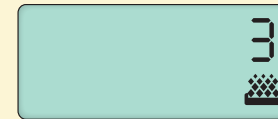


### Pre-heating the Humidifier without Pressure

Press button to adjust heat settings (1-5). button starts therapy.

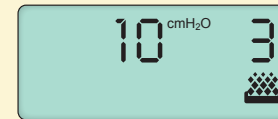

### Delivering Therapy with Humidifier


CPAP therapy pressure delivered with heated humidifier. To activate the ramp feature, press the button.

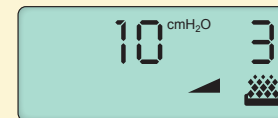

### Therapy Starting with Ramp and Humidifier

CPAP pressure gradually increasing to therapy pressure with heated humidifier.

## Use of CPAP with Battery

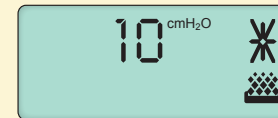

### Delivering Therapy with Humidifier on Battery

CPAP therapy pressure delivered with heated humidifier on battery.

Note: No pre-heating or heat selection options available when powered by battery.

Note: On a fully-charged battery, it may take a short period of time for the unit to recognize battery and display symbol.

To adjust ramp settings, see ramp instructions. To begin therapy, press .

## CPAP Symbols



Connected to power source



Battery power



Up and down selection arrows



Pressure ramp



Heated humidifier

# Quick Set-up Guide for *clinicians*

866-722-2507

[www.aeiomed.com](http://www.aeiomed.com)



everest<sup>2</sup>

The everyday CPAP that  
can also go mobile



AEIOMed<sup>®</sup>

AEIOMed, Inc.  
1313 5th Street SE, Suite 205  
Minneapolis, MN 55414  
[www.aeiomed.com](http://www.aeiomed.com)  
Toll free: 866-722-2507  
Phone: 612-455-0550  
Fax: 612-455-0551  
Email: [info@aeiomed.com](mailto:info@aeiomed.com)



PN101159 Rev A


Everest and AEIOMed are registered trademarks of AEIOMed, Inc.  
© 2008 AEIOMed, Inc. All rights reserved.



AEIOMed<sup>®</sup>


# Set-up Instructions

To begin, plug in the power connector while depressing the  and  buttons

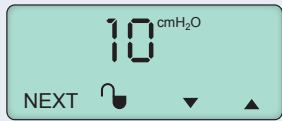




## Therapy Pressure

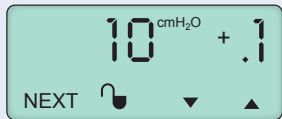Use  or  arrows to adjust pressure. Press NEXT to continue.






## Pressure Calibration

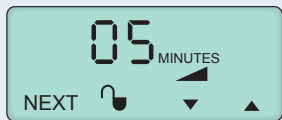Blower powers on, use manometer to check pressure. Use  or  arrows to fine tune delivered pressure. When finished, press NEXT.






## Ramp Time

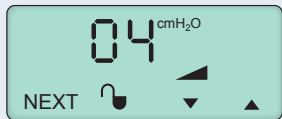Use  or  arrows to adjust ramp time in 5 minute increments (5-45 min). If 0 minutes, NO RAMP. Press NEXT to continue.




## Starting Ramp Pressure

Only appears if >0 minutes selected. Use  or  arrows to adjust pressure. Press NEXT to continue.

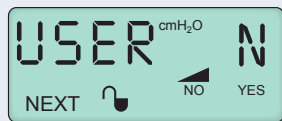




## Starting Ramp Pressure User Adjustable? Y/N

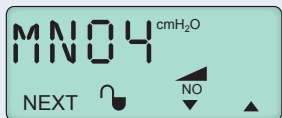Select YES or NO. If NO, next screen will not appear. Press NEXT to continue.



## Minimum Starting Ramp Pressure

Use  or  arrows to select lowest user adjustable ramp pressure. Press NEXT to continue.

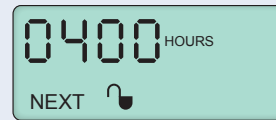


## Total Compliance Hours

Recorded total user hours. Press NEXT to continue.




## Compliance Sessions $\geq 8$

Number of on/off sessions equal to or greater than 8 hours each. Press NEXT to continue.

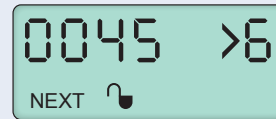


## Compliance Sessions $\geq 6$

Number of on/off sessions equal to or greater than 6 hours each. Press NEXT to continue.

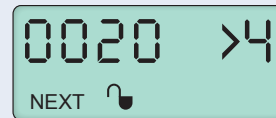

## Compliance Sessions $\geq 4$

Number of on/off sessions equal to or greater than 4 hours each. Press NEXT to continue.

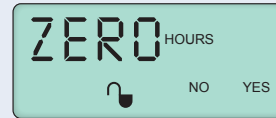


## Zero Compliance Hours and Compliance Sessions? Y/N

Select YES or NO. Selecting YES will delete all recorded hours and sessions.




## Sure? Y/N

If YES was selected this double checks the intent to delete hours. Select YES or NO.

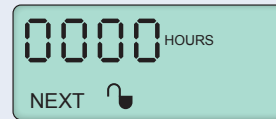


## Total Compliance Hours

Shows the result of the previous selection. Press NEXT to continue.

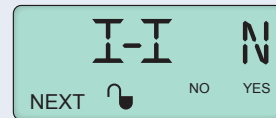


## Interface Interrupt\*

Select YES or NO. Select YES, this feature sends CPAP into stand-by mode if mask is removed for 60 seconds or more. Select NO, CPAP runs continuously. Refer to Clinician Manual for more information. \*Setting Interface Interrupt to NO will result in compliance reporting based on blower hours.

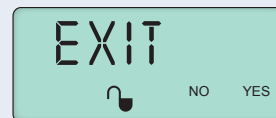

## Exit Clinician Mode? Y/N

Select YES to save set-up information and return to stand-by mode. Select NO to go through set-up again. Changes will not be saved unless YES is selected.

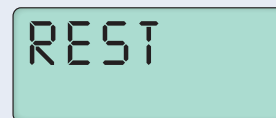

## Return to Stand-by Mode

The Everest<sup>®</sup>2 is ready for patient use.

# CPAP Symbols



Connected to power source



Battery power



Up and down selection arrows



Pressure ramp



Heated humidifier