



**DirectHome  
MEDICAL**

Presented With Compliments From  
[www.DirectHomeMedical.com](http://www.DirectHomeMedical.com)

**AEIOMed®**  
Making CPAPs for Life



Quick Set-up Guide  
for  
**Patients**



**AEIOMed®**

1313 5th Street SE, Suite 205 | Minneapolis, MN 55414  
Toll Free 866.722.2507 | Phone: 612-455-0550 | Fax: 612-455-0551  
Email: [info@aeiomed.com](mailto:info@aeiomed.com)

Disclaimer: This promotional brochure is not meant for use as an instruction manual or directions for using a CPAP. Please refer to the user manual packed with your Everest® CPAP for complete details related to using this device.

PN101160 Rev A

Headrest®, Everest® and AEIOMed® are registered trademarks of AEIOMed, Inc. The Everest® 3 and HeadRest® are covered by U.S. patents 6,854,465, 7,089,941, D555,235 and other U.S. and foreign patents issued and pending.


**Your Everest® CPAP – The Go Anywhere CPAP**

## Set-up Instructions

To begin, plug in the power connector


### Ramp Settings and Compliance Sessions

for use of CPAP with or without heated humidifier

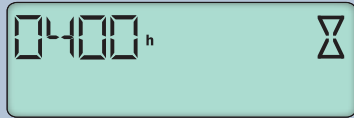


#### Total Compliance Hours

Momentarily displays total hours of use before going into stand-by mode. CPAP must be in stand-by mode to adjust ramp or humidifier settings.

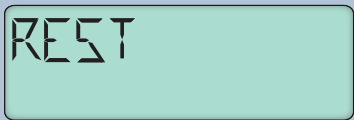


#### Stand-by Mode

Ready to begin operation. Press button.

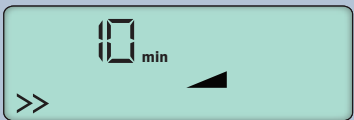


#### Ramp Time Period

Time period to reach therapy pressure (set by clinician). Press >> to continue.

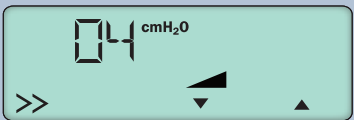


#### Starting Ramp Pressure

Use or arrows to adjust starting ramp pressure. If arrows are not present, this feature has not been prescribed for you. Press NEXT to continue.

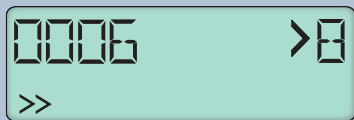


#### Compliance Sessions $\geq 8$

Number of sessions equal to or greater than 8 hours. Press >> to continue.

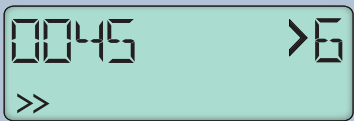


#### Compliance Sessions $\geq 6$

Number of sessions equal to or greater than 6 hours. Press >> to continue.

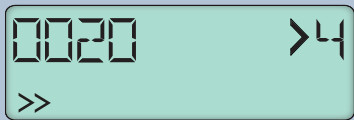

#### Compliance Sessions $\geq 4$

Number of sessions equal to or greater than 4 hours. Press >> to continue.

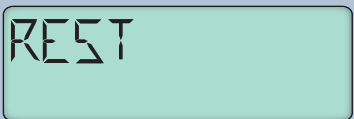

#### Stand-by Mode

Ready to begin operation. To begin therapy, press . To adjust humidifier settings, see following instructions.

## Humidifier Settings

for use of CPAP with or without heated humidifier




#### Stand-by Mode with Humidifier Reservoir in Place

Ready to begin operation with humidifier in place. Press button.




#### Pre-heating the Humidifier without Pressure

Press button to adjust heat settings (1-5). button starts therapy.

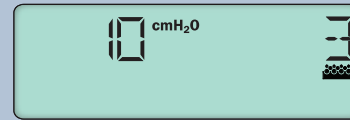

#### Delivering Therapy with Humidifier


CPAP therapy pressure delivered with heated humidifier. To activate the ramp feature, press the button.

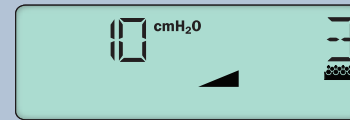

#### Therapy Starting with Ramp and Humidifier

CPAP pressure gradually increasing to therapy pressure with heated humidifier.

## Use of CPAP with Battery

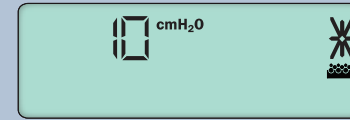

#### Delivering Therapy with Humidifier on Battery

CPAP therapy pressure delivered with heated humidifier on battery.

Note: No pre-heating or heat selection options available when powered by battery.

Note: On a fully-charged battery, it may take a short period of time for the unit to recognize battery and display symbol.

To adjust ramp settings, see ramp instructions. To begin therapy, press .

### CPAP Symbols



Connected to power source



Battery power



Up and down selection arrows



Pressure ramp



Heated humidifier