
SleepStyle™ 600

CPAP SERIES

Thermo
Smart™

Operating Manual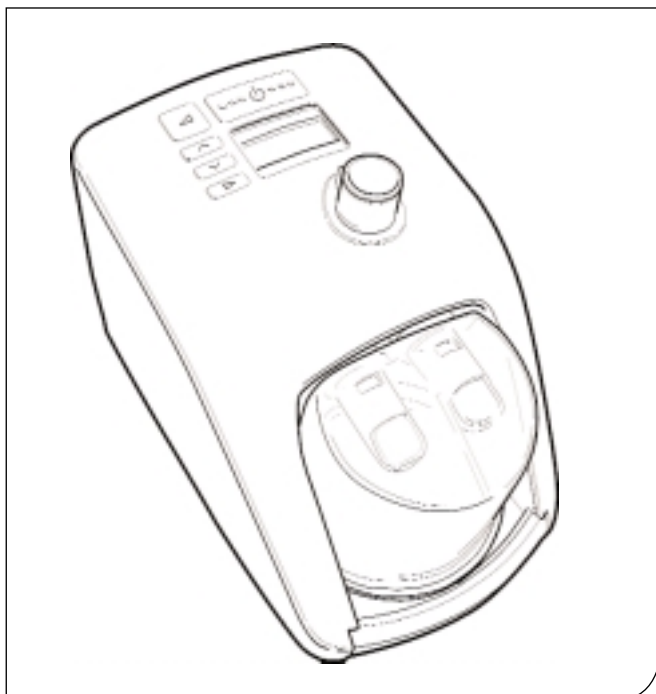

Presented With Compliments From
www.DirectHomeMedical.com

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Manuals per l'utente

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Fisher & Paykel
HEALTHCARE

SECTION

English · · · · · A

Français · · · · · B

Español · · · · · C

Italiano · · · · · D

Nederlands · · · · · E

Deutsch · · · · · F

SleepStyle™ 600

CPAP SERIES

HUMIDIFIED CPAP SYSTEMS

The SleepStyle™ 600 CPAP Series humidified CPAP system is a range of CPAP with integrated heated humidification and a heated breathing tube. It can be used in the sleep lab and at home.

For further assistance, please contact your local F&P office – see reverse for addresses. Please keep this manual for future reference.

TABLE OF CONTENTS

1. Symbol Definitions	A 2
2. Warnings	A 2
3. Oxygen Use Warnings	A 2
4. Intended Use	A 2
5. Description of the device	A 3
5.1 Package Contents	A 3
5.2 Important Parts of the device	A 3
5.3 Accessories	A 3
5.4 ThermoSmart™ Technology	A 4
6. Cleaning & Maintenance	A 5
7. Setup Instructions	A 5
8. Controls and Display	A 6
8.1 Display Description and Functions	A 6
8.2 Patient Menu Summary	A 7
8.3 Patient Menu Additional Settings Summary	A 7
9. Frequently Asked Questions	A 8
10. Product Specifications	A 8
11. Operating conditions	A 8
12. Storage and Transport conditions	A 8
13. Troubleshooting	A 8
14. Appendix	A 9

PLEASE READ ALL INSTRUCTIONS BEFORE INITIAL USE
Caution: USA Federal Law restricts this device for sale by or on the order of a physician.

1. SYMBOL DEFINITIONS

Type BF
Applied Part

ATTENTION
Consult
accompanying
documents

IPX1
Drip Proof

Alternating
Current

Class II
Double Insulated

Power
On/Off

CE 0123

93/42/EEC
Class IIb

2. WARNINGS

PLEASE NOTE:- This manual refers to the SleepStyle™ 600 CPAP Series unit as "the device".

1. The device should only be used with masks, connectors and delivery tubes recommended by Fisher & Paykel Healthcare, or by your physician or respiratory therapist.
2. Never operate the device if:
 - a) it has a damaged power cord or plug, b) the heated breathing tube has been damaged with holes, tears or kinks, c) it has been dropped or damaged, d) it has been dropped into water, e) it is not working properly, f) it is connected to a PC during CPAP treatment.Return the device to an authorised service center for examination and repair.
3. Do not use the device when the room temperature exceeds 35°C (95°F) as this may lead to an elevated airflow temperature which may give rise to an irritation of the airways.
4. Ensure an air filter is fitted when operating your unit.
5. Remove the humidification chamber before filling. Do not fill water through the heated breathing tube connection.
6. Do not fill the chamber with boiling water.
7. When handling the device with a chamber and water avoid tilting the machine to prevent any chance of water entering the unit enclosure.
8. Place your device on a level surface lower than your head height to prevent water entering the respiratory tubing should the device be knocked over.
9. Do not store or use your device where it can fall or be pulled into water. If water has entered the unit enclosure, disconnect the power cord and discontinue use. Seek advice from your homecare provider.
10. Never block the air openings of the device or place it on a soft surface such as a bed or couch, where the filter area may be blocked. Keep the air openings free of lint, hair or the like.
11. Ensure that the unit is dry before plugging into the power socket.
12. A mask should not be used unless the device is switched on, and operating properly. The vent hole(s) associated with the mask should never be blocked.
13. Due to the operating temperature of the heater plate, avoid making contact to the exposed heater plate or chamber base.
14. At low CPAP pressures, the flow through the mask may be insufficient to clear all exhaled gas from the tubing, and consequently some re-breathing may occur.
15. Adjusting the pressure control can result in ineffective therapy. The pressure control must only be adjusted by a healthcare professional.
16. Never drop or insert any object into any opening or heated breathing tube.
17. Always switch off the device and unplug the power cord from the power socket before cleaning.
18. Empty all the water from the chamber before transporting your device.
19. To remove the heated breathing tube, always push the sleeve up to avoid tube damage.
20. Refer all maintenance to qualified service personnel except as outlined in this manual.

3. OXYGEN USE WARNINGS

If supplemental oxygen is used with the device, the oxygen flow must be turned off when the device is not operating. This ensures that oxygen will not flow through the heated breathing tube and accumulate in the device enclosure.


- The device should be located in a position where ventilation around the device is not restricted.
- It is recommended that supplemental oxygen be administered at the mask.
- Oxygen should not be used while smoking or in the presence of an open flame.
- The use of oxygen in therapy requires that special care be taken to reduce the risk of fire. Any materials which will burn in air and some that will not are easily ignited and burn rapidly in high concentrations of oxygen. Accordingly, for safety it is necessary that all sources of ignition be kept away from the product and preferably out of the room in which it is being used.
- A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances must be kept away from oxygen regulators, cylinder valves, tubing and connections, and all other oxygen equipment.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection and leak rate.

4. INTENDED USE


The device is used to assist with patient breathing while sleeping for the purpose of treating Obstructive Sleep Apnea (OSA). This is done by the delivery of Continuous Positive Airway Pressure (CPAP) in order to prevent airway obstruction. The addition of heated respiratory humidification to the device relieves the drying and irritating effects on the patient airways which usually arises from use of a CPAP system. The device is for use on adult patients at home or in the sleep lab. Please refer to the HC600-Series product range listed in the appendix section of this manual.

5. DESCRIPTION OF THE DEVICE

5.1 Package Contents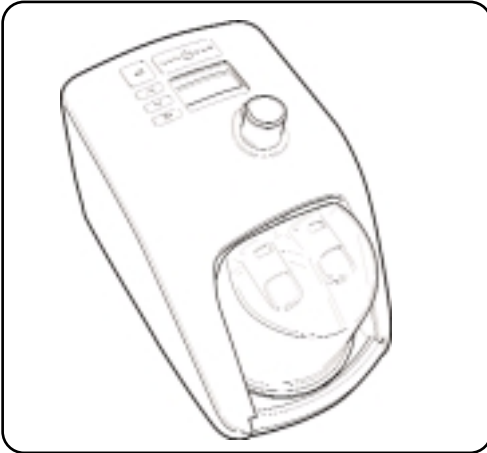


1 x SleepStyle™ 600 CPAP Series

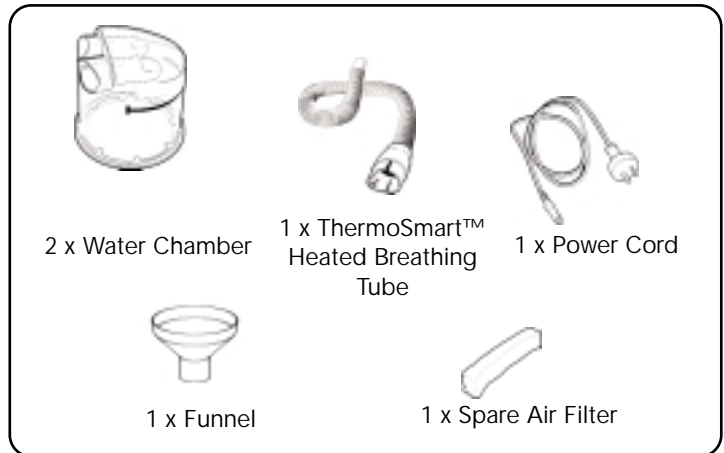Accessories

5.2 Important Parts of the Device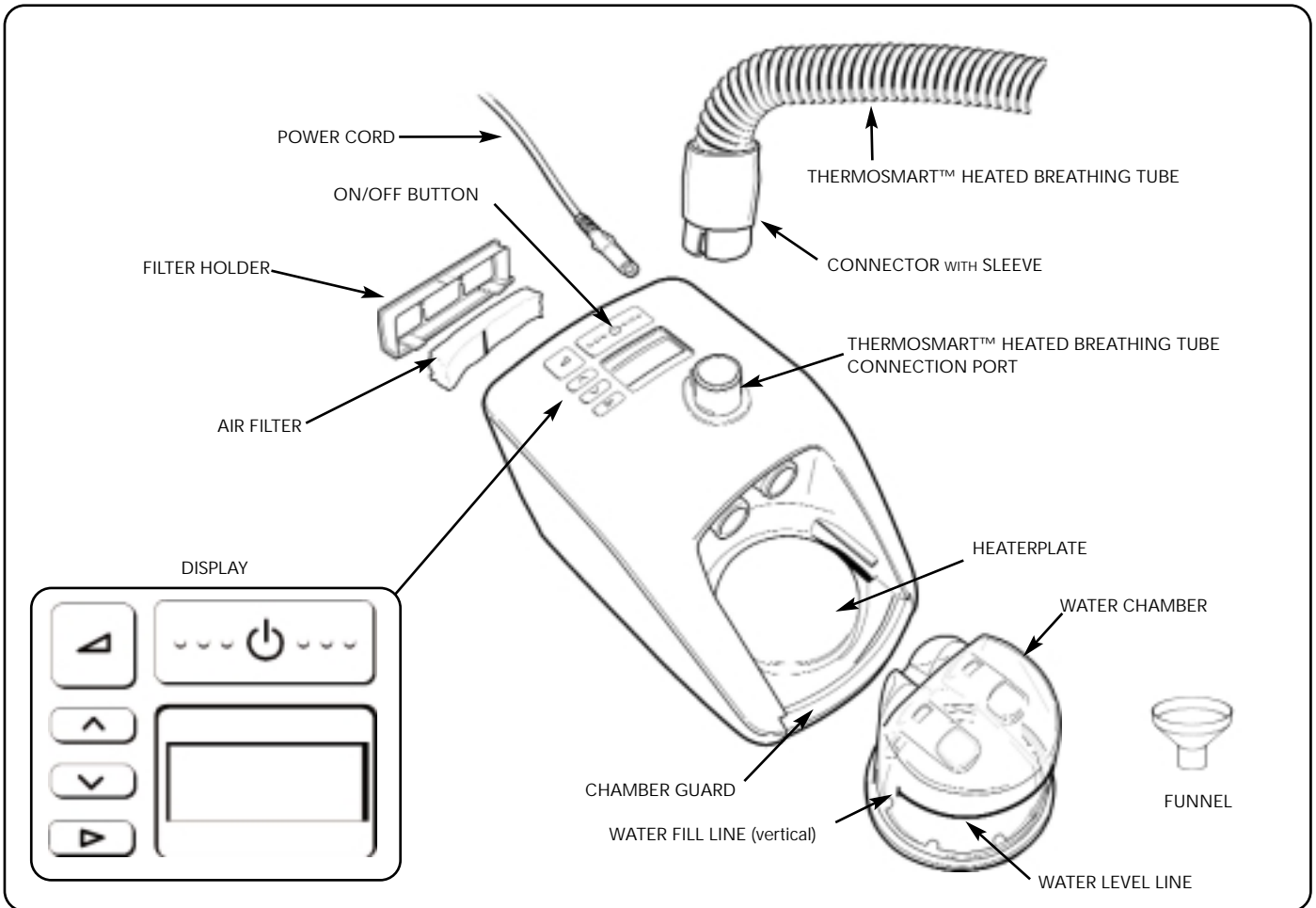

5.3 Accessories

HC365	Water Chamber – disposable	900HC230	Compliance Maximizer Software
HC360*	Water Chamber – reusable	900HC232	Compliance Download Serial Cable
900HC522	ThermoSmart™ Heated Breathing Tube	HC225	Remote Control (for use in sleep lab)
900HC240	Filter (2-Pack)	900HC226	Lead Extension Kit for HC225
900HC604	Outlet Cleaning Kit HC600		*Not available in all countries

5.4 ThermoSmart™ Technology

This device has been designed for optimal comfort. It offers the unique heated breathing tube technology ThermoSmart™. ThermoSmart™ technology allows for the delivery of optimal (higher and customized) levels of humidity to the patient, independent of ambient temperature.


This device can be adjusted on two levels to achieve optimal comfort:

- ThermoSmart™ Comfort setting regulates humidity and temperature output. It can be adjusted from 1 to 4 with increments of 0.5. The device is preset to 2.5 which is suitable for most patients.
 - If you experience dryness or find the air too cool: Increase the setting
 - If you find there is too much moisture in the tube or the air feels too hot: Decrease the setting
 - If you do not achieve optimal comfort with this setting you need to alter your ThermoSmart™ Climate setting.
- ThermoSmart™ Climate setting offers an extended range of temperature and humidity combinations. The ThermoSmart™ Climate setting has a range of 1 to 3 with increments of 1. The device is preset to 2 which is suitable for most patients. ThermoSmart™ Climate setting 3 is recommended for oral CPAP delivery.
 - If you feel the range of settings offered by the ThermoSmart™ Comfort setting are too dry or too cool: Increase the ThermoSmart™ Climate setting
 - If you feel the range of settings offered by the ThermoSmart™ Comfort setting are too warm or too humid: Reduce the ThermoSmart™ Climate setting

To operate ThermoSmart™ Comfort and ThermoSmart™ Climate setting, refer to section 8 “Controls and Display” of this manual.

6. CLEANING & MAINTENANCE

1. Switch off the device and unplug from the power socket.
2. Wipe the exterior of the device with a clean, damp (not wet) cloth and mild dishwashing detergent. Do not use harsh abrasives or solvents, or these may damage the device.
3. DAILY
Clean chamber, heated breathing tube and heated breathing tube connection port.
 - Remove the heated breathing tube from the device by pulling sleeve upwards to unlock the connector. Then remove connector.
 - Wash the heated breathing tube with soapy water, rinse and then hang - up the heated breathing tube with its ends pointing to floor to dry.
 - Remove the chamber by pushing down the finger guard and pulling out the chamber.
 - Pour out and discard the remaining water.
 - Clean the chamber with soapy water, rinse and dry.
 - Thoroughly wipe the inside of the heated breathing tube connection port with a clean damp low lint cloth.
4. WEEKLY
Thoroughly clean the chamber .
 - Soak the inside of the chamber for 10 minutes in a solution of one part white vinegar to five parts water.
5. Replace the air filter when it becomes significantly discoloured, at least once every 3 months or after 1000 hours machine running time.
 - Take the filter holder from the back of the device and take out the filter.
 - Replace the old filter with a new filter: Ensure the vertical black line is facing towards the device.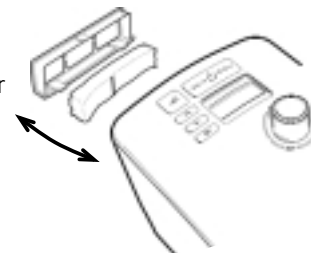

7. SETUP INSTRUCTIONS

1. Remove your device from its packaging.
2. Place the device on a low shelf or on the floor beside your bed, the unit needs to be placed below head height.
3. Remove one water chamber from the packaging.
4. Remove the blue caps and discard them (Fig.1).
5. Fill the chamber up to the water filling line with water (Fig.2)
Never fill the chamber through the heated breathing tube connector port. When moving your device, ensure that the water chamber is empty. Machine failure due to water damage is not covered by warranty.
6. To attach the water chamber to the device, press down the finger guard and slide the chamber on (Fig.3) (Fig. 4).
7. When the chamber is fitted correctly, the finger guard will click into place.
8. One end of the heated breathing tube has a grey plastic connector. Lift the sleeve and slide connector on to the device. Push the sleeve down to lock (Fig. 5), (Fig.6), (Fig.7).
9. Connect the other end of the heated breathing tube to the mask.
10. Plug the device power cord into your household power socket.
11. Switch on the device by pressing the on/off button (See Fig.8).
The current ThermoSmart™ Comfort setting is displayed.
12. For changing the setting refer to section 8 "Controls and Display".

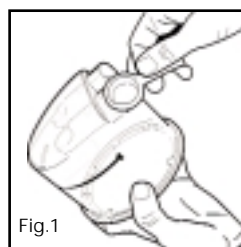
Fig.1

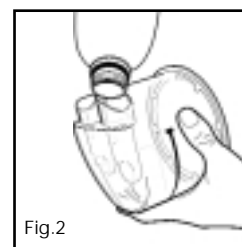
Fig.2

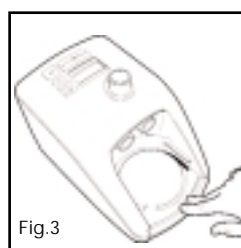
Fig.3

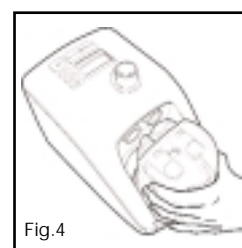
Fig.4

Fig.5

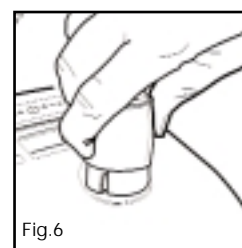
Fig.6

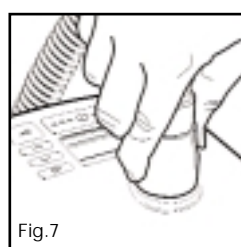
Fig.7


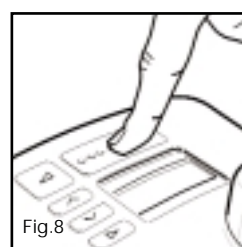
Fig.8

Your device is now ready for use.

8. CONTROLS AND DISPLAY

8.1 DISPLAY DESCRIPTION AND FUNCTIONS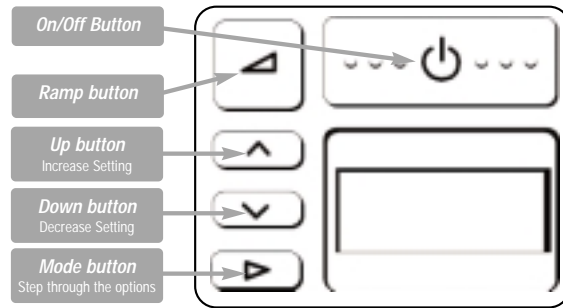


KEY	FUNCTION & DESCRIPTION	OPERATION
	ON/OFF Switches the device ON and OFF	TO SWITCH "ON" <ul style="list-style-type: none"> Press button down briefly The device will default to display the ThermoSmart™ Comfort setting. TO SWITCH "OFF" <ul style="list-style-type: none"> Remove your mask Press button briefly
	RAMP Reduces pressure to a lower level to help you fall asleep. The pressure will gradually return to full operating pressure over 20 minutes	TO SWITCH "ON" <ul style="list-style-type: none"> Press NB: to reset ramp during a cycle the device must be switched off first TO SWITCH "OFF" <ul style="list-style-type: none"> Press
 	INCREASE/DECREASE Adjusts settings up or down	<ul style="list-style-type: none"> Press the and button to raise or lower settings
	MODE To step through options	<ul style="list-style-type: none"> Press sequentially to step through and view options available

PATIENT MENU	DISPLAY	OPERATION
For viewing of CPAP settings, humidity, compliance data, altitude and pressure settings		TO VIEW ITEMS Press the "" key to sequentially view:
		<ul style="list-style-type: none"> ThermoSmart™ Comfort setting (default display) For adjusting ThermoSmart™ Comfort setting levels to minimize upper airway side effects from CPAP use Press the and button to raise or lower settings
	* hr	<ul style="list-style-type: none"> Total run hours Displays the total number of hours the device has been switched on (includes hours without mask use)
	* hr/☾	<ul style="list-style-type: none"> Average compliance hours per night Displays average number of hours the device has been used at night with the mask on
		<ul style="list-style-type: none"> Checksum Displays information for medical practitioner or dealer use to check compliance
		<ul style="list-style-type: none"> Altitude setting Displays altitude setting Press or to change altitude
	cmH ₂ O	<ul style="list-style-type: none"> Pressure setting Displays pressure setting


PATIENT MENU - ADDITIONAL SETTINGS	DISPLAY	OPERATION
To access additional settings: From the Default setting "THERMOSMART™ COMFORT" Press the "" key for 3 seconds		<ul style="list-style-type: none"> ThermoSmart™ Climate setting To change the delivered temperature: Press for warmer more humid air delivery. Press for cooler and drier air delivery.** Refer to section 5.4
		<ul style="list-style-type: none"> Altitude Select "m" for metres or "ft" for feet using the and keys
	3000	<ul style="list-style-type: none"> Altitude level <ul style="list-style-type: none"> - Press to change altitude level - Set the altitude level by pressing the and keys - Press to exit menu
	cmH ₂ O	<ul style="list-style-type: none"> Actual Pressure Displays actual pressure NOTE: Only available on devices with Automatic Altitude Adjustment/Leak Compensation feature

- **Warning:** Failure to select correct altitude setting (for any given location) will have an adverse effect on delivered pressure.
- * only available on devices with compliance monitoring.
- ** When the ThermoSmart™ Climate setting is altered the ThermoSmart™ Comfort setting will revert back to default setting 2.5.

8.2 PATIENT MENU SUMMARY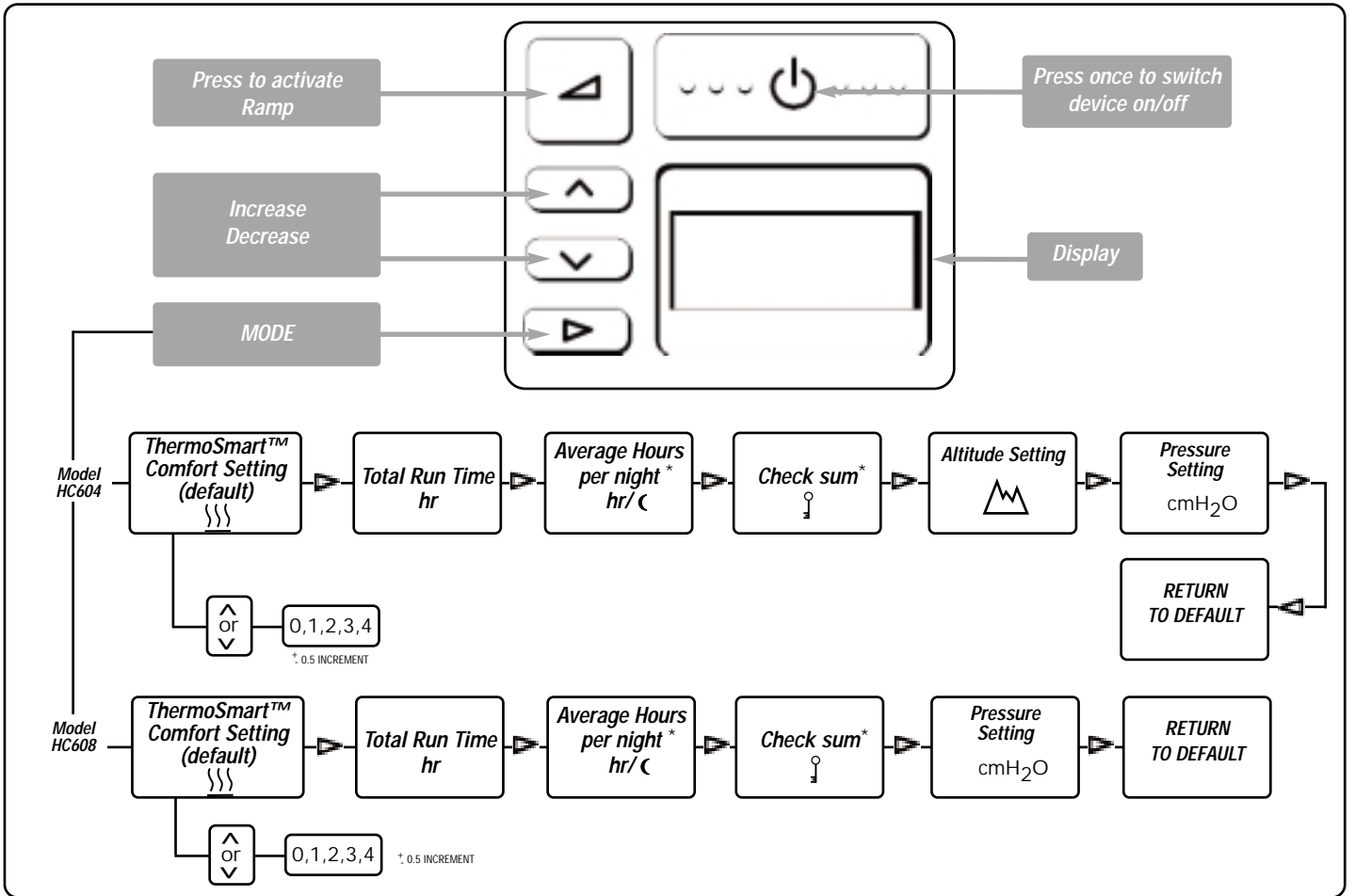

8.3 PATIENT MENU - ADDITIONAL SETTINGS SUMMARY

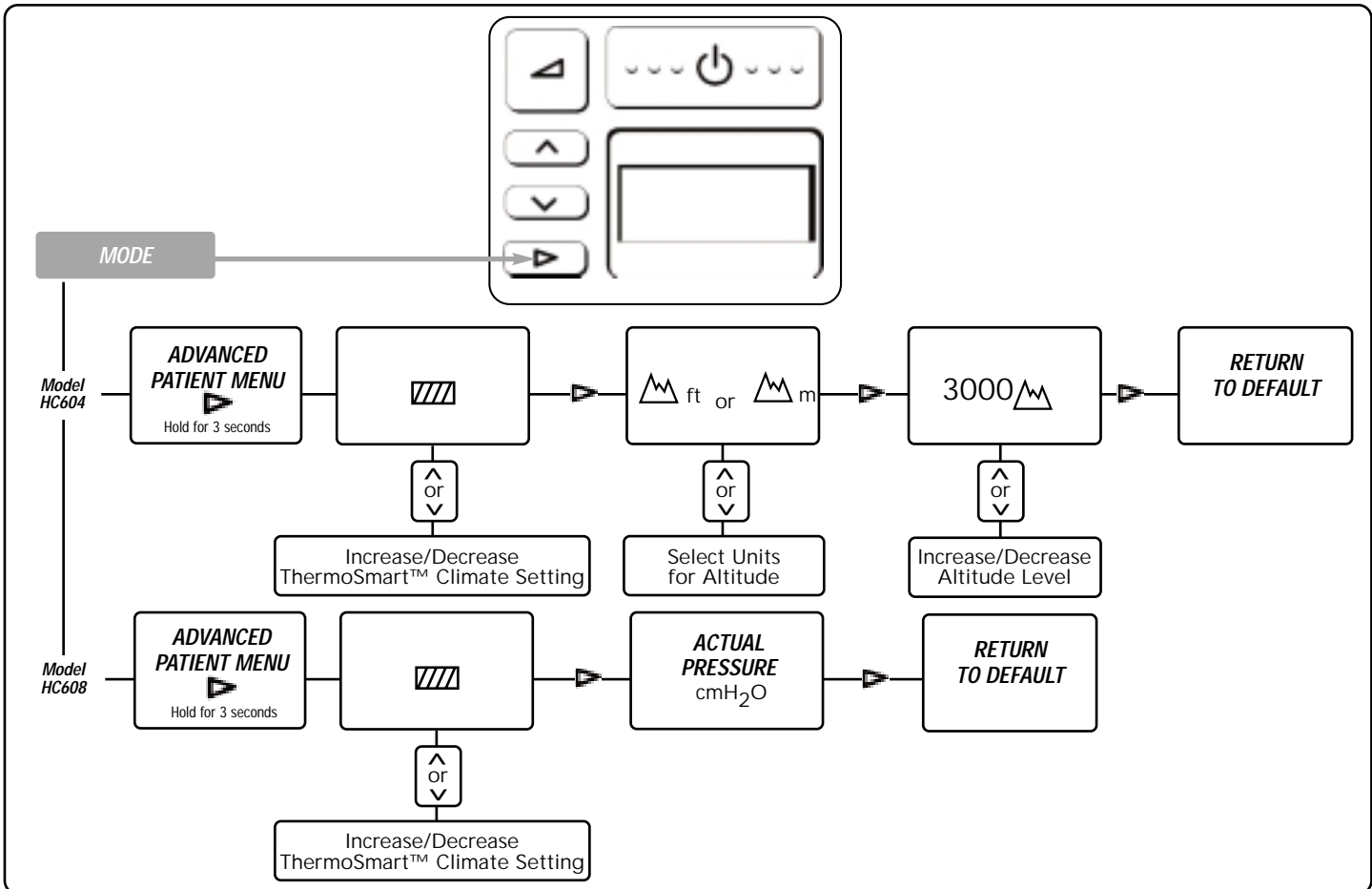- **Warning:** Failure to select correct altitude setting (for any given location) will have an adverse effect on delivered pressure.
- * only available on devices with compliance monitoring.

9. FREQUENTLY ASKED QUESTIONS

Do I have to use distilled water with my device?

The use of distilled water will maximize the life of the water chamber. Water from the faucet will often contain minerals which can damage the chamber, causing pitting in the base, even if the water is passed through a filter.

Can I use my device in other countries?

Yes. Simply use the appropriate electrical socket adaptor and the device will automatically adapt to any voltage supply.

Note: When moving your device, ensure that the water chamber is empty. Machine failure due to water damage is not covered by warranty.

Can I use supplemental oxygen with my device?

Yes, oxygen can be administered at the mask. Turn the device on before turning on the oxygen. Ensure that the oxygen source is turned off before turning off the device to avoid oxygen accumulating in the machine.

When do I replace my water chamber?

Replace the chamber if the plastic walls of the chamber become cracked or cloudy or the chamber base becomes pitted. (Note: If the chamber leaks at all, it should be replaced immediately).

Can I use my device without a heated breathing tube?

Yes you can, but you will not gain the optimum humidity performance the device offers.

The device will then not display the heated breathing tube symbol and show the following display:

10. PRODUCT SPECIFICATIONS

DIMENSIONS:	273mm x 170mm x 175mm (10.7" x 6.7" x 6.9")
WEIGHT:	2.1kg (4.6 lbs) unit only 3.4kg (7.5 lbs) packaged in bag incl. accessories
ELECTRICAL RATINGS	
Supply Frequency:	50-60 Hz
Supply Voltage and Current:	100-115V (1.5A, max 1.7A), 220-240V (0.75A, max 2.0A) Note: These values represent average current
DC to AC Inverter requirements:	115V - 200W/300W surge 230V - 300W/500W surge
Heater Plate :	85W max
Heater Plate Temperature:	65°C (149°F) max
Heated breathing tube Power Setting:	30W max
PERFORMANCE	
Pressure Range	4 to 20 cmH ₂ O
Altitude Range	0 to 3000m, or 0 to 9000 ft
Warm-up time:	30 min

STANDARDS COMPLIANCE

Designed to conform to the requirements of:

IEC 60601-1
UL60601-1
CSA C22.2/No. 601.1
AS3200.1.0
EN60601-1

The device complies with the electromagnetic compatibility requirements of IEC 60601-1-2. In certain circumstances, the device may affect or be affected by nearby equipment due to the effects of electromagnetic interference. If this should happen, try moving your unit or the location of the device causing interference, or alternatively consult your homecare provider.

11. OPERATING CONDITIONS:

AMBIENT TEMPERATURE:	5° - 35°C
HUMIDITY:	10 - 95% RH
ALTITUDE:	0 - 3000m (9000ft)

Above 1500m (4500 ft) the maximum operating pressure will be reduced.

12. STORAGE AND TRANSPORT CONDITIONS:

The device should be stored and transported in environmental conditions of: -10° to 60°C .

13. TROUBLESHOOTING:

If you feel that your device is not operating correctly, consult your homecare dealer.

Do not attempt to repair the device yourself.

Fisher & Paykel Healthcare have a policy of continued product improvement and reserve the right to alter specifications without notice.

14. APPENDIX

The following table lists the main difference in features across the HC600 Product Range.

MODEL	Fully Integrated CPAP with Heated Humidification	Patient Compliance Monitoring (LCD & Download)	Auto Altitude Adjustment/ Leak Compensation
HC604	✓	✓	-
HC608	✓	✓	✓

- NOTE:
- All fully integrated models with ThermoSmart™ technology
 - Some models may not be available for sale in certain countries

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