

TRANSCEND[®]

Portable Solar Battery Charger[™]

INTENDED USE


The Transcend Portable Solar Battery Charger[™] is an accessory that charges the Transcend P4 and Transcend P8 Batteries, which power Transcend Sleep Apnea Therapy Devices (Transcend Auto[™], Transcend EZEX[™] and Transcend[®]).



Presented With Compliments From
www.DirectHomeMedical.com

DIRECTIONS FOR USE

1. Unstrap the Velcro from the Solar Charger[™].
2. Unfold the Solar Charger to its maximum size.
3. Place the Solar Charger in direct sunlight with the solar panels facing up.
Note: The LED may illuminate red depending on Ambient light conditions.
4. Plug the Transcend P8 Battery or Transcend P4 Battery into the solar charger port.
5. The solid red LED indicates that the battery is charging. Once the battery is fully charged, the solid green LED will illuminate. Typical charge times for fully depleted batteries are 6 hours for the P4 Battery and 12 hours for the P8 Battery, respectively. Charge times may vary depending on UV Index.
6. When charging is complete, fold the Solar Charger and strap the Velcro into place. Do not fold or crease the solar panels, creasing may destroy their function.

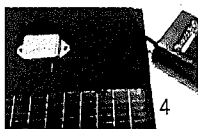

CLEANING AND MAINTENANCE

To clean the Solar Charger, use a dampened cloth to wipe away any dirt or residue. No further maintenance is required.

WARNINGS

Do not allow the Battery temperature to exceed 113°F (45°C). When charging in hot, full sun conditions, as in a desert environment, protecting the Battery from direct sunlight is advised. The Battery can be protected by placing it under the solar charger or covering it with a cloth. Do not fold or crease the solar panels, creasing may destroy their function.

Do not attempt to connect the Battery to AC power or use the Transcend Mobile Power Adaptor[™] while the Battery is connected to the Solar Charger.

CAUTIONS

The P4 and P8 Batteries are not waterproof. Take caution to keep them dry. If rain is imminent, or if charging is to take place in a wet or moist environment, take steps to keep the Battery dry, or simply take it inside.

The Solar Charger is water resistant and can withstand rain or other typical environmental conditions. Do not store the Solar Charger wet, because mold may develop. Be sure to dry the device thoroughly before storing.

Technical Specifications	
Power	13-14W
Dimensions	12.5" x 7" x 1.25"
Weight	0.7lbs (317.5g)
Storage Temperature Range	-20°-60°C (-4°-140°F)
Operating Temperature Range	0-45°C (32-113°F)

WARRANTY

PowerFilm, Inc. Limited Product Warranty

PowerFilm, Inc., the parent company of PowerFilm® Solar, warrants that this PowerFilm® Solar product is free from defects in materials and workmanship that impact performance for a period of one (1) year. The duration of the Limited Warranty is from the date of initial purchase.

The foregoing warranty does not apply to any defect, failure or damage caused by improper use, unauthorized modifications, neglect, improper testing, attempt to repair, inadequate maintenance, or damage from accident, fire, abuse or other hazard. Please visit the product specific section below for details on the power output warranty as well as any other product specific limitations. PowerFilm's sole obligation and liability for products failing to comply with this warranty shall be, at PowerFilm's choice, either to repair the products or replace the defective product with a new or remanufactured equivalent at no charge, provided that the warranty procedures have been followed.

PowerFilm will return repaired or replacement product at PowerFilm's cost.

Disclaimer of Warranties, EXCEPT AS EXPRESSLY SET FORTH IN THIS AGREEMENT, THE PRODUCTS ARE PROVIDED ON AN "AS IS" BASIS AND POWERFILM MAKES NO WARRANTY IN CONNECTION WITH THE SUBJECT MATTER OF THIS AGREEMENT AND HEREBY DISCLAIMS ANY AND ALL IMPLIED OR STATUTORY WARRANTIES, INCLUDING ALL IMPLIED WARRANTIES OF TITLE, NON INFRINGEMENT, QUIET ENJOYMENT, ACCURACY OF DATA, MERCHANTABILITY AND FITNESS FOR A PARTICULAR USE OF PURPOSE REGARDING SUCH SUBJECT MATTER. To the extent that a party may not, as a matter of applicable law, disclaim any implied warranty, the scope and duration of such warranty shall be the minimum permitted under such law.

Foldable and Rollable Solar Chargers

Power Output Warranty: PowerFilm warrants to the original purchaser that under standard PowerFilm test conditions the PowerFilm Solar product will produce 80% of the minimum rated output for the term of three (3) years. If PowerFilm determines that the PowerFilm® Solar product is not providing the warranted percentage of its specified minimum power output during the term of the warranty, PowerFilm will, at its sole option, repair the product, replace the product with a new or remanufactured equivalent, or provide a pro-rated discount on the purchase of replacement product.

To obtain service under this warranty, contact Somnetics customer service at rma@somnetics.com or call 651-621-1800 with the 11 digit serial number, description of the failure, contact information, and a return shipping address. After providing the necessary information Somnetics will issue a return material authorization number (RMA) and provide further detail for return.

Manufactured by Powerfilm for Somnetics International, Inc



SOMNETICS, INTERNATIONAL, INC.
33 5TH AVE NW, SUITE 500, NEW BRIGHTON, MN 55112
WWW.MYTRANSCEND.COM



AUTHORIZED REPRESENTATIVE (MDD 93/42/EEC)
METHODIZE EUROPE
DRIEBLAD 11



NL-2811 EG REEUWIJK
THE NETHERLANDS
TEL. +31 6 2129 4150