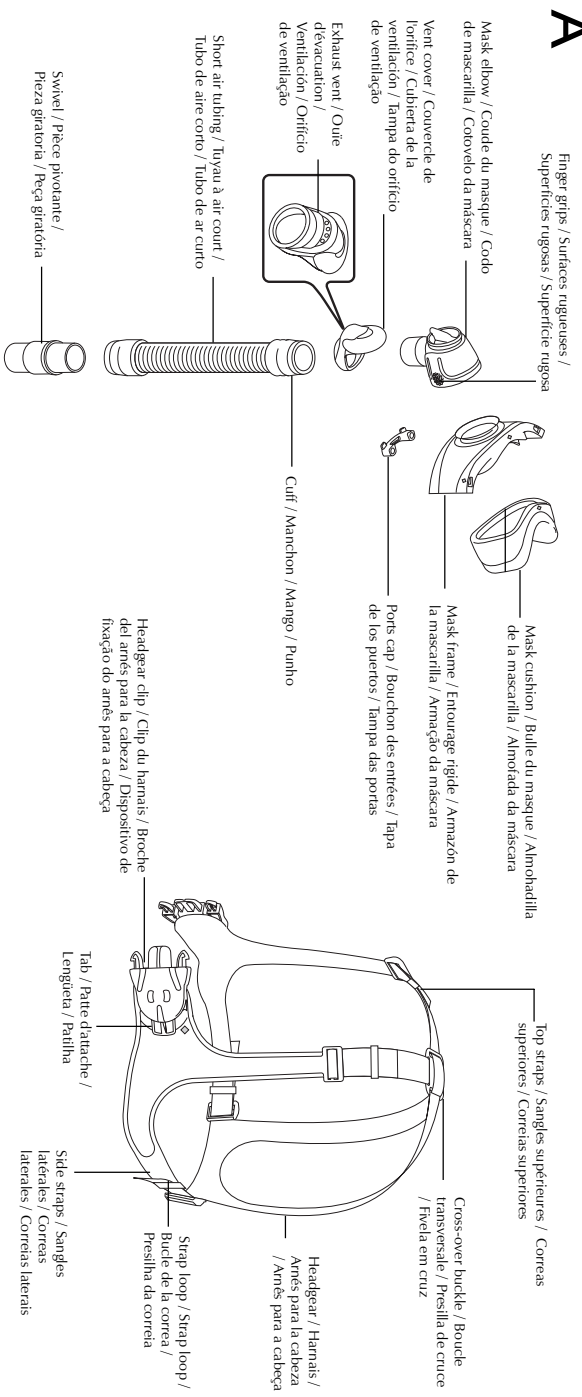
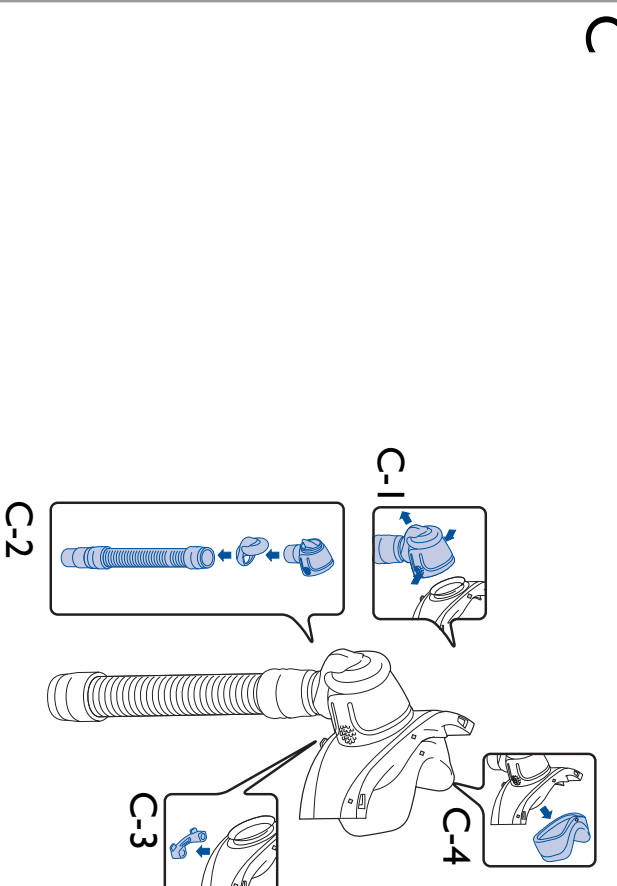
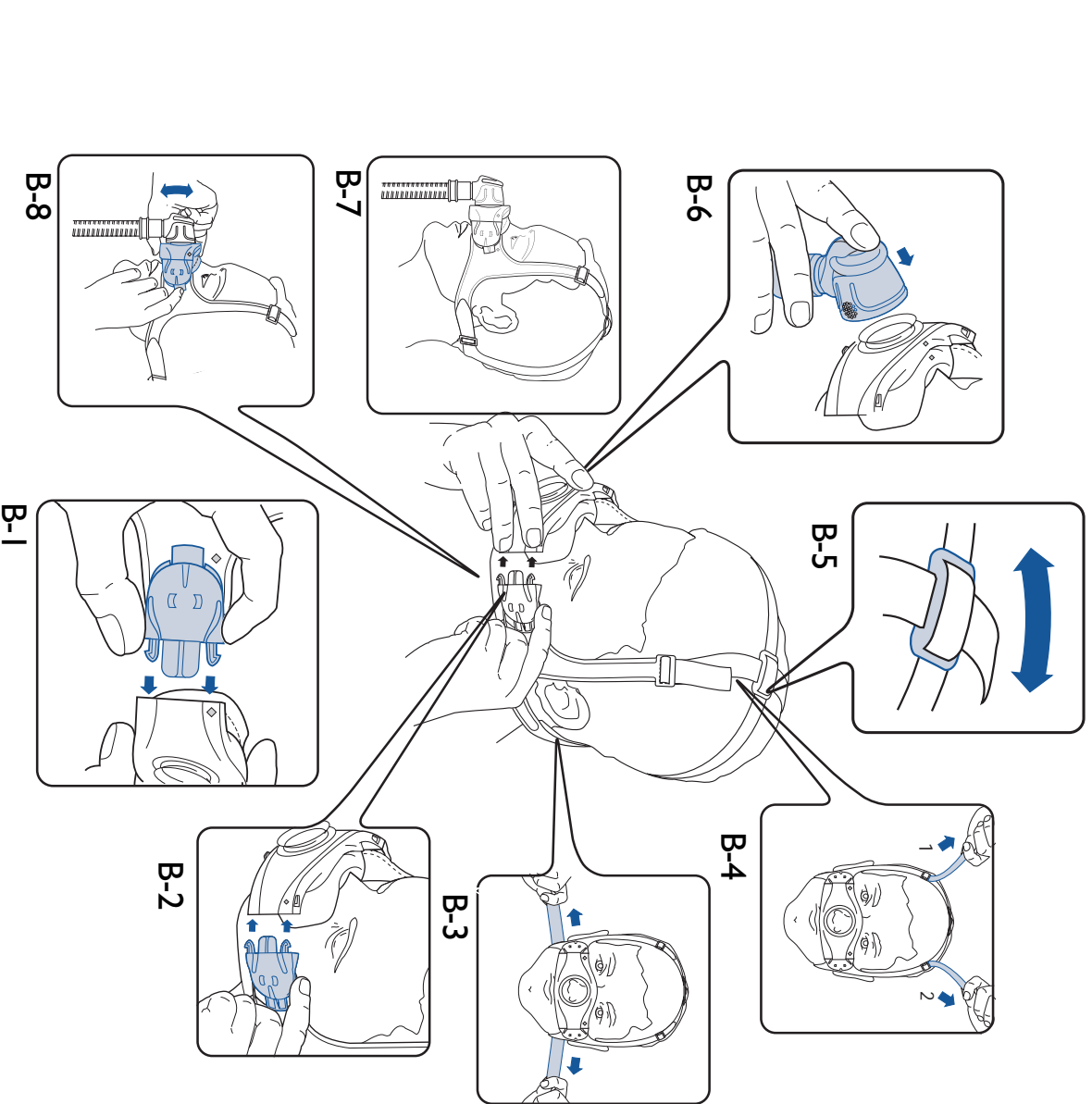
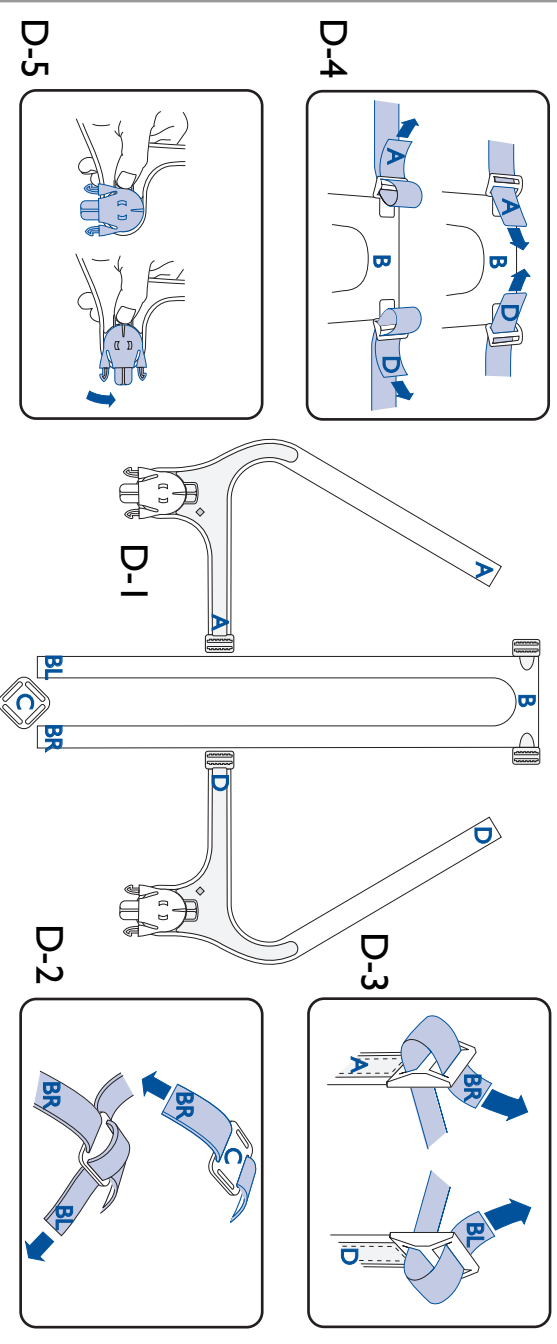
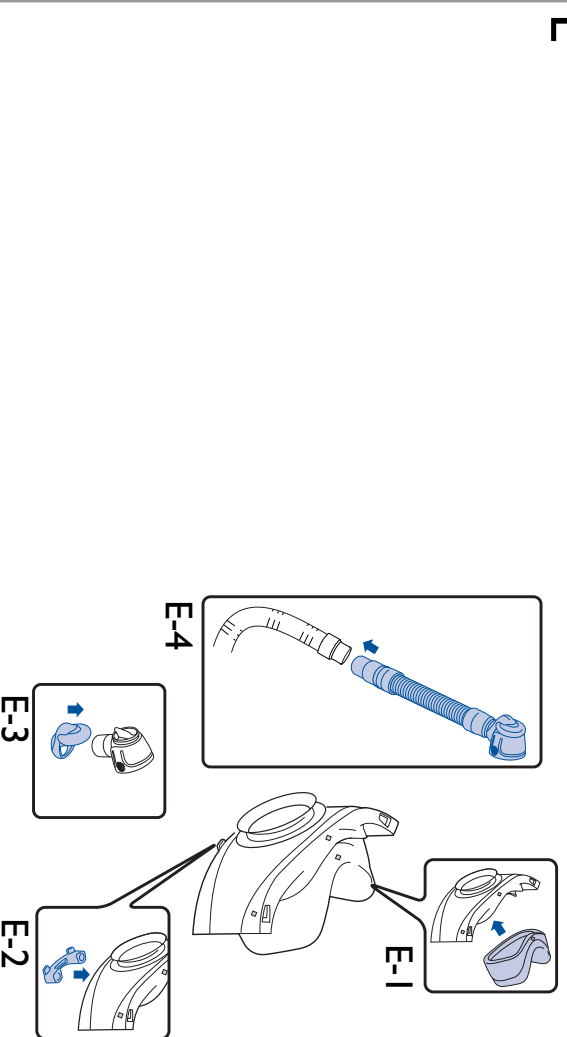
**A****C****B****D****E**

**Manufactured by:**

**ResMed Ltd** 97 Waterloo Road North Ryde NSW 2113 Australia  
Tel: +61 (2) 9886 5000 or 1 800 658 189 (toll free) Fax: +61 (2) 9878 0120 Email: reception@resmed.com.au

**ResMed Offices:**

**ResMed Corp** 14040 Danielson Street Poway CA 92064-6857 USA  
Tel: +1 (858) 746-2400 or 1-800-424-0737 (toll free) Fax: +1 (858) 746-2900 Email: reception@resmed.com

**ResMed (UK) Ltd** 65 Milton Park Abingdon Oxfordshire OX14 4RX UK  
Tel: +44 (1235) 862 997 Fax: +44 (1235) 831 336 Email: reception@resmed.co.uk

**ResMed Asia Pacific Ltd** 97 Waterloo Road North Ryde NSW 2113 Australia  
Tel: +61 (2) 9886 5000 or 1 800 991 900 (toll free) Fax: +61 (2) 9889 1471 Email: reception@resmed.com.au

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**US DESIGNATED AGENT:** ResMed Corp

**EU AUTHORISED REPRESENTATIVE:** ResMed (UK) Ltd

[www.resmed.com](http://www.resmed.com)

**Mirage Vista Mask**

Protected by patents: AU 712236, AU 724360, US 6439230. Other patents pending.  
Mirage, Vista and Mirage Vista are trademarks of ResMed Ltd.

© 2003 ResMed Ltd.

Re-order number: 60804/2 03 11

**RESMED**

Presented With Compliments From  
[www.DirectHomeMedical.com](http://www.DirectHomeMedical.com)

MIRAGE® VISTA™ MASK

USER'S  
GUIDE

MODE D'EMPLOI

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[www.resmed.com](http://www.resmed.com)

# RESMED

## MIRAGE® VISTA™ MASK

The MIRAGE® VISTA™ nasal mask is intended for adult patients prescribed continuous positive airway pressure (CPAP) and bilevel therapy in hospital, clinic, and/or home environments.

### CAUTION

In the USA, federal law restricts this device to sale by, or on the order of, a physician.

### MEDICAL INFORMATION

**Note:** *The mask system does not contain latex materials. If you have ANY reaction to any part of the mask system, discontinue use and consult your sleep therapist.*

### WARNINGS

- This mask should be used only with CPAP or bilevel systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or bilevel system is turned on and operating properly. The exhalation device should never be blocked.

#### **Explanation of the Warning:**

CPAP and bilevel systems are intended to be used with special masks with connectors which have vent holes or separate exhalation devices to allow continuous flow of air out of the mask. When the CPAP or bilevel machine is turned on and functioning properly, new air from the machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP or bilevel machine is not operating, enough fresh air will not be provided through the mask, and

exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP and bilevel systems.

- The MIRAGE VISTA features an elbow that can be easily removed while the mask is fitted. The mask should not be used with the elbow attached unless the CPAP or bilevel system is turned on and operating properly.
- If oxygen is used with this CPAP or bilevel machine, the oxygen flow must be turned off when the CPAP or bilevel machine is not operating.
- Please refrain from smoking while oxygen is in use.

#### **Explanation of the Warnings:**

When the CPAP or bilevel device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP or bilevel machine enclosure. Oxygen accumulated in the CPAP or bilevel machine enclosure will create a risk of fire. This warning applies to most types of CPAP or bilevel machines.

**Note:** *At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and leak rate.*

### CAUTION

At low CPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.

**Explanation of the Caution:** The exhaust vent flow is lower at lower mask pressures (refer Pressure/Flow curve). To avoid rebreathing it is recommended that a mask pressure of 4 cm H<sub>2</sub>O or more be used.

## PARTS OF THE MASK

See **Figure A**.

## FITTING THE MASK

### Figure B

- 1 Attach one of the headgear clips to one side of the mask frame. To do this, push the headgear clip into the slot on the mask frame until it clicks into place (**Fig. B-1**).

**Note:** Ensure that you line up the two diamond-shaped markers (◇) on the mask frame and adjacent to the headgear clip.

- 2 Place the mask over your nose. The top of the mask cushion should sit over the middle of your nose.
- 3 Place headgear over head, with lower straps below ears. Hold the mask frame over the nose and connect the unattached headgear clip (**Fig. B-2**).
- 4 Symmetrically tighten side straps at back of head so they are comfortable, but not too tight (**Fig. B-3**). Evenly tighten top straps one at a time (**Fig. B-4**).

**Note:** Strap loops are provided to hold the free end of the headgear straps in place.

- 5 Position cross-over buckle on top of head so that side Stabilizing Panels are flat against the sides of the face (**Fig. B-5**). You may need to readjust the top straps.
- 6 Connect one end of the main air tubing to your flow generator and the other end to the swivel.

**Note:** The swivel, short air tubing and mask elbow are supplied assembled.

- 7 Turn your flow generator on.
- 8 Connect the mask elbow to the mask frame. To do this, firmly push the mask elbow onto the mask frame until it clicks into place (**Fig. B-6**).

Check that you have connected the mask elbow correctly by rotating it. Be careful not to squeeze the sides of the elbow. If the mask elbow is fitted correctly, it should rotate freely and not separate from the mask frame.

- 9 The fitted mask should look like **Fig. B-7**.
- 10 Adjust the angle of the mask by pressing on the tabs on the headgear clips and tilting the mask up or down (**Fig. B-8**) until you feel a comfortable, even pressure around your nose. The mask can be tilted at five different positions.

Example: If your headgear is not over tightened and you are feeling excessive pressure on the middle of your nose, tilt the mask down. Alternatively, if you are feeling excessive pressure on your upper lip, tilt the mask up.

- 11 If you feel air leaks around the cushion, reposition the mask on your face to achieve a better seal.

Take time to adjust the headgear straps and the position and angle of the mask to achieve a comfortable and secure fit.

**Note:** Select "Standard" as the mask option, when using the Mirage Vista with ResMed AutoSet® devices.

## REMOVING THE MASK

To remove the **MIRAGE VISTA** squeeze one of the headgear clips. This releases one side of the headgear from the mask. Pull the mask and headgear away from your head.

**Note:** Do not pull the headgear off your head without releasing the headgear clips first.

### TIP!

You can detach the elbow from the mask frame while wearing the mask. This is useful when therapy needs to be stopped for a short time; for example if you need to go to the bathroom.

## DISASSEMBLING FOR CLEANING

- 1 Detach the mask elbow. To do this, squeeze the sides of the mask elbow at the finger grips while pulling it away from the mask frame (**Fig. C-1**).
- 2 Disconnect and separate the main air tubing, short air tubing, mask elbow and vent cover (**Fig. C-2**).

### Notes

To prevent damage to the short air tubing, disconnect it by holding it at the cuff.

The swivel does not need to be disconnected from the short air tubing.

- 3 Detach the headgear from the mask frame by squeezing and releasing both headgear clips.

**Note:** The headgear does not normally need to be disassembled. If disassembled, see section on "Reassembling the Headgear".

- 4 Remove the ports cap from the mask frame (**Fig. C-3**).
- 5 Firmly pull the mask cushion away from the mask frame (**Fig. C-4**).

## CLEANING THE MASK

**Note:** The mask system (excluding the headgear, swivel and short tube) is validated for multipatient use. Clinicians should refer to the *MIRAGE VISTA Disinfection / Sterilization Guide* for details.

### WEEKLY

Hand wash the headgear in warm, soapy water. Use pure soap. Rinse well and allow it to air dry out of direct sunlight.

**Note:** The headgear may be washed without being disassembled.

### DAILY/AFTER EACH USE

Hand wash the mask components in lukewarm, soapy water. Use pure soap. Rinse the components well and allow them to air dry out of direct sunlight.

## PRECAUTIONS

- Do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers antibacterial agents, or scented oils to clean any part of the mask system or air tubing. These solutions may cause damage and reduce the life of the product.
- Do not expose any part of the mask system or tubing to direct sunlight as it may deteriorate.

## REASSEMBLING THE HEADGEAR

### Figure D

When disassembled, the headgear has four parts:

A: Left strap; B: Middle strap; C: Cross-over buckle; D: Right strap

- 1 Lay out the four parts of the headgear as shown in **Figure D-1**.
- 2 Following **Figure D-2** thread the straps BR and BL through the cross-over buckle (part C).
- 3 Thread strap BR through the buckle on Part A. Thread strap BL through the buckle on Part D (**Fig. D-3**).
- 4 Thread the straps of part A and part D through the buckles on part B (**Fig. D-4**).
- 5 With the headgear clip facing down, fit the headgear clip onto the hole on the headgear. While pressing the headgear tab down, rotate the headgear clip away from the tab to lock it into place (**Fig. D-5**). Repeat this with the other headgear clip.

## REASSEMBLING THE MASK

### Figure E

- 1 Insert the mask cushion into the groove of the mask frame (**Fig. E-1**). Fit the top section first and then work your way around, pushing the mask cushion into the mask frame.


**Note:** The diamond markers on the mask frame and on the mask cushion (one on top, two at the bottom) must line up.

- 2 Push the ports cap onto the mask ports (**Fig. E-2**).
- 3 Attach the vent cover to the mask elbow. To do this, insert the elbow into the retaining ring of the vent cover. Push the vent over the front of the mask elbow (**Fig. E-3**). Ensure that the lip of the vent cover is securely seated in the groove of the elbow.
- 4 Connect the short air tubing to the mask elbow.
- 5 Connect the swivel to main air tubing ready for next use (**Fig. E-4**).

## TECHNICAL SPECIFICATIONS

### PRESSURE / FLOW CURVE

The following graph shows the rate of airflow through the mask vents at varying mask pressures.
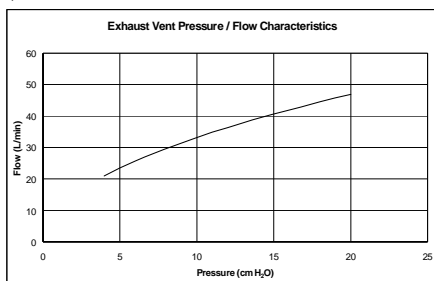
### DEAD SPACE INFORMATION

The dead space of the mask is provided below for both cushion sizes:

- (1) Standard 84 mL
- (2) Deep 84 mL

**Explanation:** Dead space is the empty volume of the mask up to the elbow.

## REPLACEMENT PARTS

Part No.	Item
60918	Mirage Vista Headgear Assembly
60919	Mirage Vista Headgear Clip (2 pk.)
60920	Mirage Vista Headgear Clip (10 pk.)
60921	Mirage Vista Cushion, Standard
60922	Mirage Vista Cushion, Deep
60924	Mirage Vista Frame
60925	Mirage Vista Elbow Assembly
60926	Inlet Tube with Swivel Assembly (1 pk.)
60927	Inlet Tube with Swivel Assembly (10 pk.)
60928	Mirage Vista Frame System
16570	Ports Cap (2 pk.)
16571	Ports Cap (10 pk.)

## LIMITED WARRANTY

ResMed warrants that your ResMed mask system, including its components: mask frame, cushion, headgear, tubing and other accessories ("Product") shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

For further details refer to the Warranty.

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