

Mirage™ SoftGel

NASAL MASK

Fitting Guide



1

- Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



2

- Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.



3

- Reconnect the lower headgear clip, ensuring it clicks into place.



4

- Adjust the upper straps evenly. Do not over-tighten.



5

- Adjust the lower straps to a comfortable position.



6

- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.



TIP

- Adjust dial to correct leak at nose.



TIP

- Adjust bottom straps to correct leak at top lip.



Presented With Compliments From
www.DirectHomeMedical.com

Mirage™ SoftGel

NASAL MASK

ConvertAble™ Comfort

The Mirage™ SoftGel Nasal Mask offers exceptional comfort with ResMed's renowned performance and quality. Easy and quick to fit, Mirage SoftGel is part of ResMed's exclusive ConvertAble™ Series, giving you the choice of two

mask cushions on one durable frame. Featuring a soft touch with excellent support, Mirage SoftGel adds the comfort, durability and freedom of choice that only a ConvertAble mask can offer.

MicroFit dial
offers 24 positions for better fit, seal and comfort

ResMed ConvertAble frame
accommodates both Mirage SoftGel and Mirage Activa™ LT cushions for business efficiency, cost savings and flexibility

360° elbow rotation
positions the tubing for maximum comfort


Quiet, diffused vent design
disperses air softly and quietly, away from bed partner

Elbow partition
aids CO₂ washout and reduces noise output

SoftGel's unique DoubleGel™ cushion
Clear gel layer provides ultimate comfort; blue gel layer provides support and stability

Set-and-forget headgear clips
retain optimal headgear settings; easy to attach and detach

Swivel
offers a convenient way to connect and disconnect air tubing

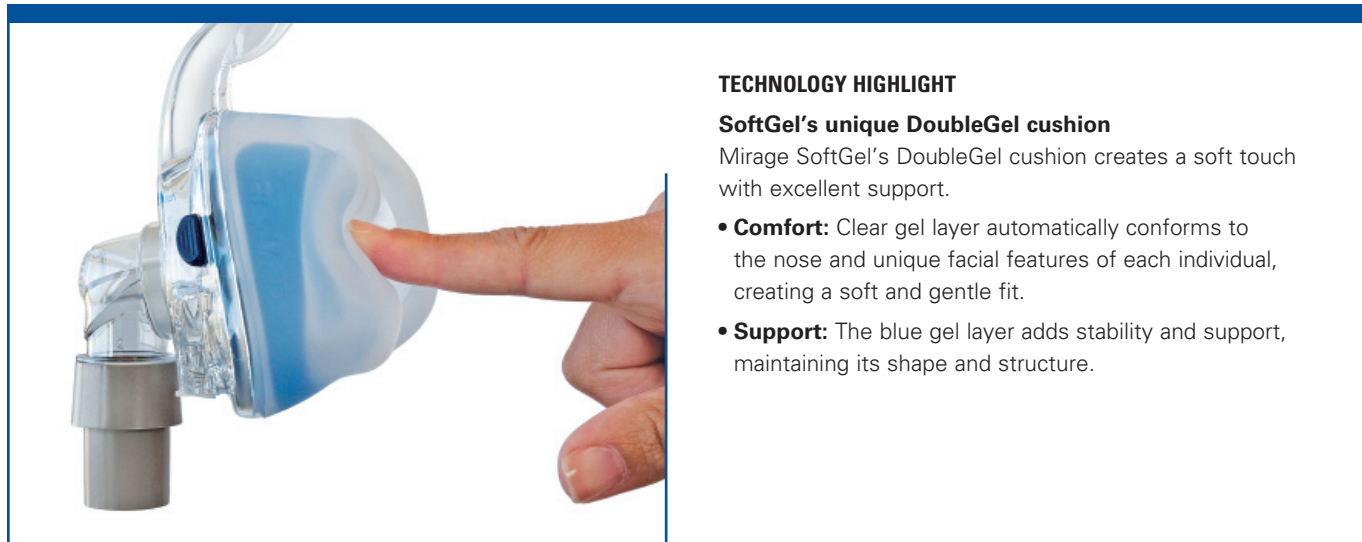

TECHNOLOGY HIGHLIGHT

SoftGel's unique DoubleGel cushion

Mirage SoftGel's DoubleGel cushion creates a soft touch with excellent support.

- **Comfort:** Clear gel layer automatically conforms to the nose and unique facial features of each individual, creating a soft and gentle fit.
- **Support:** The blue gel layer adds stability and support, maintaining its shape and structure.

USER BENEFITS

Unparalleled softness and comfort

- The DoubleGel cushion's outer layer provides unparalleled softness. The blue gel layer maintains stability and support
- Fitting is a simple task—for both current users and new patients

Ultra-quiet therapy

- Special array of small, built-in vents diffuse air quietly. That means a good night's sleep for patients and their bed partners

Correct fitting made easier

- Combination of the Mirage dual-wall cushion with the DoubleGel cushion design makes fitting simple
- The cushion adapts to individual facial contours, creating a comfortable fit with ease

ConvertAble mask means freedom of choice

- ResMed ConvertAble masks offer the choice of two different cushions on one frame
- Freedom of choice helps patients accept their therapy early and stay engaged in treatment

PROFESSIONAL BENEFITS

ConvertAble means inventory efficiencies

- Softgel is a multi-patient, multi-use mask suitable for repeated use in the lab
- Interchangeable and shared components promote inventory efficiencies and cost savings
- Fitting and training time is reduced
- Mirage SoftGel is extremely durable, ensuring consistent performance and fewer callbacks

Customer satisfaction and added compliance

- Increased therapy acceptance through first-time fit—long-term comfort
- Interchangeable cushions improve patient compliance while avoiding the cost of expensive mask replacements
- Choice of comfort and fit ensures patient satisfaction—minimizes callbacks
- Comprehensive user support package includes travel bag and user guide with mask fitting and care instructions

PATIENT-PREFERRED

- 8 out of 10 patients rate SoftGel better in comfort and durability than other gel masks¹
- 3 out of 4 patients rated their sleep better with SoftGel¹



PRODUCT CODES

Mirage SoftGel Nasal Mask

Medium	61601
Large	61602

PRODUCT CODES

Reimbursement Codes

Nasal mask use with positive airway pressure device, 1 per 3 months	A7034
Replacement cushion for nasal mask, 2 per month	A7032
Headgear used with positive airway pressure device, 1 per 6 months	A7035

Mirage™ SoftGel

NASAL MASK

Components Card

Complete system

M 61601 L 61602

Frame system*

(no headgear)

61627 (M)
61628 (L)

Frame assembly*


(no headgear, no cushion)

60175 (St)

Elbow assembly

(elbow, swivel)

16399

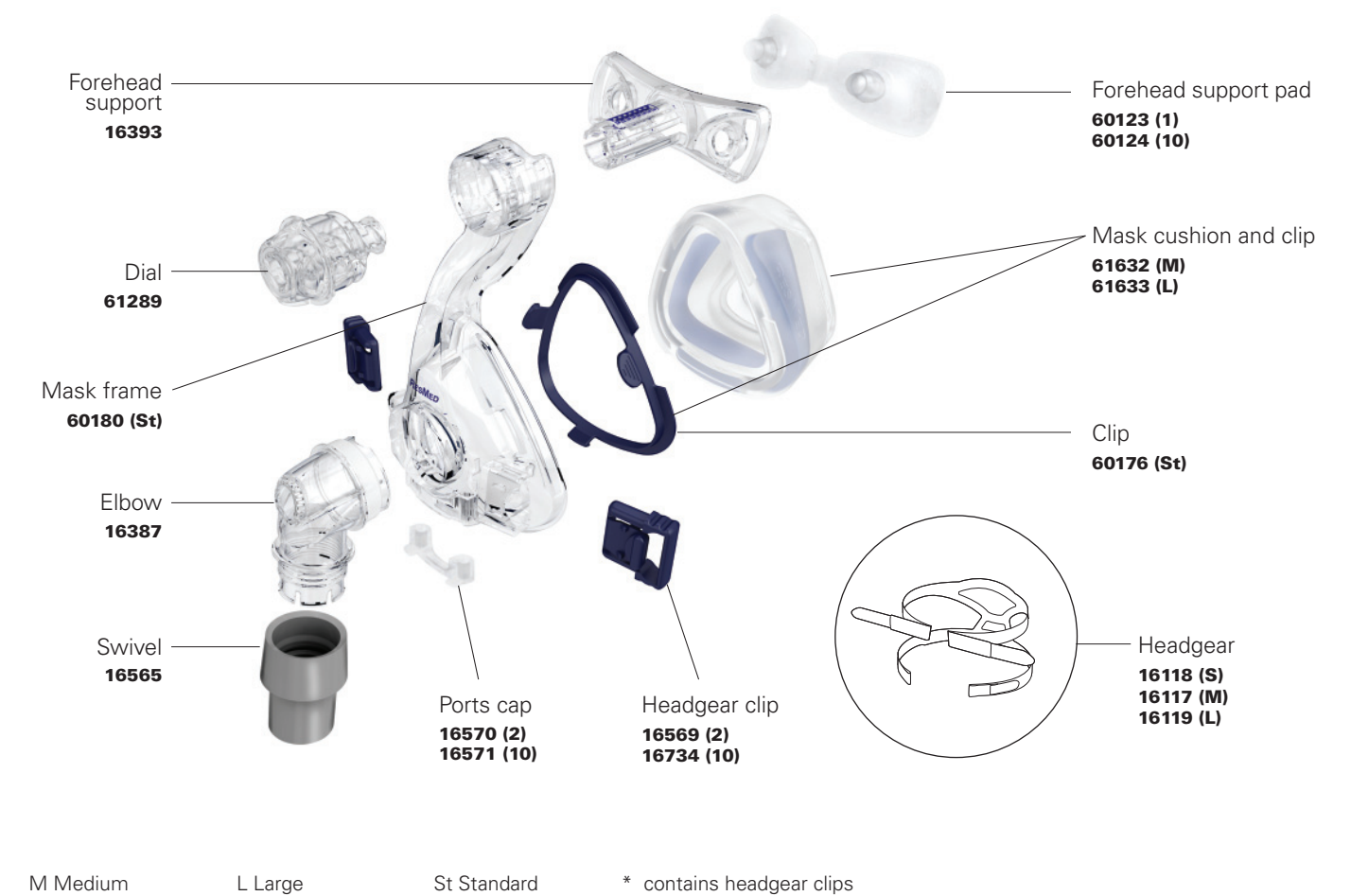

⚠ For details, refer to the *Mirage SoftGel Nasal Mask User Guide*.

¹ ResMed market research study of gel masks.