

PHANTOM FITTING TIPS

The nose clip bends the mask shell to fit nose bridge



1. The nose clip can bend to seal at the bridge of the nose. If the clip has been shaped previously, return to the original shape by hooking your thumbs in the corners of the mask opening and bending the corners out. The cushion opening should resemble an even triangle.
2. Assemble the mask to the headgear, as indicated in product instructions. Place on patient, center the headgear on head, and assist with strap adjustments. If possible, fit patient in supine or preferred sleep position. Connect mask tubing to airway pressure device tubing, and turn unit on. Begin at a low pressure setting.
3. Instruct the patient to bend the nose clip to fit their nose bridge if they feel an air leak. Gradually increase air pressure to prescribed level.
4. If necessary, pull the mask away from face and reset on nose; move mask higher or lower; readjust headgear. The mask tubing can be secured to headgear crown or draped to the patient's side (as in illustrations below).



Presented With Compliments From
www.DirectHomeMedical.com

GENERAL INFORMATION

- 90 day warranty against manufacturer's defects
- Wash on a daily basis with mild soap and water
- CE Mark 0123; latex free, hypoallergenic
- Small tubing with swivel connector and quick release included with each mask
- Negligible pressure drop throughout mask system
- Compatible with all positive airway pressure devices

SleepNet
Corporation

SleepNet Corporation · 5 Merrill Industrial Drive, Hampton, NH USA 03842
1-603-758-6600 · Toll Free (800) 742-3646 USA and Canada
Fax 1-603-758-6699 · www.sleep-net.com