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PHANTOM[®] Nasal Mask

CONTENTS: **Phantom**[®] Nasal Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The Sleepnet Corporation **Phantom**[®] Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H₂O for the treatment of adult obstructive sleep apnea.

GENERAL INFORMATION - WARNINGS AND CAUTIONS

CAUTION: U.S. Federal Law restricts this device for sale by or on the order of a physician.

CAUTION: Do not expose the **Phantom**[®] Nasal Mask to temperatures above 122°F (50°C).

CAUTION: This mask is intended for single patient use only. It may be cleaned and used repeatedly on the same person, but must not be used on multiple persons. Do not sterilize or disinfect.

WARNING: At low baseline pressure, the flow through the vent hole may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

WARNING: This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask vent hole should never be blocked. The vent hole allows a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent hole. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

WARNING: If oxygen is used with this CPAP mask, the oxygen flow must be turned off when the CPAP machine is not operating. Explanation of the warning: When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire.

WARNING: No smoking or open flames such as candles may be used when oxygen is in use.

WARNING: Do not use the **Phantom**[®] Nasal Mask with baseline pressure less than 3 cm H₂O or greater than 30 cm H₂O.

WARNING: Consult manufacturer for proper O₂ enrichment tubing.

GENERAL INFORMATION

The **Phantom**[®] Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. The mask has an expected service life of 6 to 12 months. Please dispose of the mask and accessories in accordance with local and regional guidelines. The **Phantom**[®] Nasal Mask and its accessories are Latex Free.

CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

NOTE: Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tears or punctures.

NOTE: The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and above recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new **Phantom**[®] Nasal Mask with care.

TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 0.65 cm H₂O; at 100 L/min: 1.78 cm H₂O

A-weighted Sound Pressure Level at 1 m distance: 32.7 db (A); A-weighted Sound Power Level: 40.7 db (A)

User instructions for 2 Point Headgear

1. Attach tubing to mask. The tubing and mask plug can be switched from right or left.
2. Thread the horizontal straps through the slots in the mask shell.

3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.

4. Attach tubing to headgear with loop strap, if preferred.

NOTE: The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.

6. Gently bend nose clip to create a comfortable fit and reduce leaks.

REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.

8. Open one of the horizontal straps and detach from the mask.

 Phantom[®] is a trademark of Sleepnet Corporation; Patent #6,019,10.

Visit our website at www.sleep-net.com.

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CE₀₁₂₃ **Phantom®** ist ein Markenzeichen der Sleepnet Corporation. Patent #6,019,10.