

# EverGo™ Quick-Start Guide



Step 2

## 1. Attach the carrying handle and shoulder strap to the carrying case.

## 2. Insert the batteries.

To insert the two lithium ion batteries, unzip the top of the unit, lift the top of the bag and open the cover.

Pull back the gray retainer tab on the top of the EverGo and gently insert one battery with the Respironics label facing the inside of the device. The retainer will snap into place over the battery. Insert the second battery using the same process. Zip the bag closed.

**Important:** If either battery extends above the top gray surface, it may be in backwards. Remove it and insert it correctly.



Step 3

## 3. Place your EverGo in a well-ventilated location with the particle filter in place in the small mesh zippered compartment.

## 4. Charge the batteries using the AC Power Cord.

First, firmly connect the power cord to the power supply box. Then connect the output cable of the AC power supply to the EverGo power inlet on the side of the device. Next, plug the AC power cord into an electrical outlet.

**Important:** Before using it for the first time, you must fully charge the EverGo batteries (4 hours minimum) using the AC power cord. Do not operate it on battery power until it is fully charged.



Step 4

## 5. Connect a nasal cannula to the oxygen outlet on top of your device.

## 6. Turn on your EverGo.

Gently touch the screen anywhere. Then touch the On/Off symbol within 5 seconds or the screen will turn off.



Step 5



Step 7

## 7. Set the flow.

Touch the large number in the center of the screen (the 2 setting is shown here). Use the up/down arrows to set the flow to the rate prescribed by your physician.

## 8. Position the nasal cannula on your face and breathe normally through your nose.

## EverGo Battery Duration

The following table shows the approximate duration of use at 20 Breaths Per Minute (BPM). Times may vary based on activity level and the condition of the batteries.

Flow Setting	Breath Rate of 20 BPM (2 batteries)	Breath Rate of 20 BPM (1 battery)
1	12 hours	6 hours
2	8 hours	4 hours
3	6 hours	3 hours
4	4 hours	2 hours
5	4 hours	2 hours
6	4 hours	2 hours

## EverGo Mobile Cart Operation



### To set up and use the Mobile Cart, follow these easy steps.

1. Set the Mobile Cart on a level surface, hold the handle with one hand and open the base support.
2. Pull back the elastic strap on the back of the carrying case and slip it onto the cart handle. Then slip the accessory bag onto the cart handle.
3. Press down on the "PUSH" button located at the top of the handle. Then pull the handle upward to extend the cart arm to the desired height.
4. The cart is now ready to conveniently transport your oxygen concentrator and its accessory bag.



Resironics Inc: +1-724-387-4000  
Resironics Europe, Africa, Middle East: +33-1-47-52-30-00  
Resironics Germany: +49-8152-93 06-0  
Resironics France: +33-2-51-89-36-00  
Resironics Italy: +39-03-62-63-43-1  
Resironics UK: +44-800-1300-845  
Resironics Switzerland: +41-62-745-1750  
Resironics Asia Pacific: +852-3194-2280  
Resironics Australia: +61-2-9666-4444  
[www.resironics.eu](http://www.resironics.eu)      [www.resironics.com](http://www.resironics.com)

CAUTION: U.S. federal law restricts these devices to sale by or on the order of a physician.

Resironics is a registered trademark, and EverGo and Freedom Series are trademarks, of Resironics, Inc. and its affiliates. Covered by one or more patents. Other patents pending.

©2008 Resironics, Inc. and its affiliates. All rights reserved.

Broudy KB 06/20/08 MCI 4101484 PN 1050141