

Breeze SleepGear™

CPAP Interface System

Your First Choice in Fit and Comfort.

The *Breeze SleepGear*™ CPAP Interface System has set new standards in comfort and compliance for sleep apnea patients. Unlike traditional headgear and masks, there are no complicated or confining straps, so patients can sleep in virtually any position.

Fitting Guide for Breeze SleepGear CPAP Interface System

Step 1
Inspect the *Breeze SleepGear* interface before each use to ensure all parts are firmly connected and no visible leaks exist.



Step 2
Insert the Nasal Pillows into the nostrils, or place the *DreamSeal*® Mask over the nose.



Step 3
Place the cradle (back pad) below the curve at the back of the head.



Step 4
Place your hand on the interface spine and hold it firmly to the back of the head. Adjust the length of the *Breeze SleepGear* interface with your free hand by moving the nasal airway assembly toward the face.



Step 5
On the Nasal Pillows assembly, a new clip is available to adjust positioning of the halo straps. If using the clip, adjust by first lifting the clip and then sliding it to one of the three available positions along the hose guide that provides the most comfortable fit. Do not overtighten the halo straps.



Step 6
Adjust the height and angle of the shell to fit over the nose so the Nasal Pillows or *DreamSeal* Mask is comfortable and there are no audible leaks.



The *Breeze SleepGear* CPAP Interface System can be used with Puritan Bennett® Nasal Pillows, Nasal Dilators or the *DreamSeal* Mask, depending on your patients' needs and preferences.