

MAKERS OF  
RESPIRATORY  
VALVES SINCE 1938

# HRI



Petite = 3.6 - 3.9 inches approximately  
Extra Small = 3.9 - 4.2 inches approximately  
Small = 4.2 - 4.5 inches approximately  
Medium = 4.5 - 5.0 inches approximately  
Large = 5.0 - 5.5 inches approximately



Presented With Compliments From  
[www.DirectHomeMedical.com](http://www.DirectHomeMedical.com)

## MASK SIZING GAUGE

For Hans Rudolph, inc.  
Oro-Nasal Vmask™ and V2mask™  
Series Face Masks

**Directions:** Length measurements are used to determine the size of the Hans Rudolph Oro-Nasal mask that will best fit the patient. For measurement: the patient's facial muscles should be relaxed and jaw closed. Place the Gauge Tip to the deepest point of the nasal root depression (sellion) and note the size indication at the bottom tip of the chin. The color band width indicates the size ranges. (This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.)

**\*If between sizes, we recommend you use the smaller size.\***

**PETITE**

**EXTRA-SMALL**

**SMALL**

**MEDIUM**

**LARGE**



**HANS RUDOLPH, inc.**

MAKERS OF RESPIRATORY VALVES SINCE 1938

**ISO 9001 ISO 13485**

Tel: (913) 422-7788 USA & CANADA (800) 456-6695

Fax: (913) 422-3337 E-mail: [hri@rudolphkc.com](mailto:hri@rudolphkc.com)

[www.rudolphkc.com](http://www.rudolphkc.com)

8325 Cole Pkwy, Shawnee, Kansas 66227 USA



Printed in USA  
691143 5/06 Rev B