

Mirage Quattro™

FULL FACE MASK



- 1

 - Turn the MicroFit dial to position 24, where the forehead support is fully extended.



- 2

 - Disconnect one headgear clip, place the mask on the face and pull the headgear over the head.



- 3

 - Bring the lower straps below the ears.
 - Fasten the lower headgear clip ensuring it clicks into place.



- 4

 - Unfasten the Velcro™ tabs and pull the upper headgear straps evenly until comfortable.
 - Do not overtighten the straps.



- 5

 - Unfasten the Velcro tabs and pull the lower headgear straps evenly until comfortable.
 - Do not overtighten the straps.



- 6

 - Connect the main air tubing to the swivel.
 - Turn on the flow generator.
 - Turn the MicroFit dial until the mask is evenly and comfortably positioned on the face.
 - If any air leaks are felt, readjust the headgear.

For details, refer to the *Mirage Quattro Full Face Mask User Guide*.

ResMed Corp Poway, CA, USA +1 858 746 2400 or 1 800 424 0737 (toll free), ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). Offices in Austria, Brazil, Finland, France, Germany, Hong Kong, Japan, Malaysia, Netherlands, New Zealand, Norway, Singapore, Spain, Sweden, Switzerland, United Kingdom (see website for details). Protected by patents: AU 777033, EP 1187649, EP 1314445, HK 1057714, US 6412487, US 6439230, US 6513526, US 6581594, US 6701927, US 6796308, US 6823865, US 6823869, US 6860269, US 7021311, US 7069933. Other patents pending. Protected by design registrations: AU 309495, AU 309789, AU 309800, AU 310242, AU 311114, AU 311132, AU 311133, AU 311134, AU 311135, AU 311136, AU 311137, AU 311186, AU 311187, EU 559208. Others pending. Mirage and Mirage Quattro are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office. Velcro is a trademark of Velcro Industries B.V. ©2007 ResMed Ltd. 1011547/1 07 03