

Mirage Swift™ Nasal Pillows System

QUICK FITTING GUIDE



vent
holes

1 Place headgear on head.
Insert nasal pillows at the
entrance of the nostrils.

**Important: ensure vent
holes are facing upwards**



2 Adjust headgear straps;
first the back and then
the top.



3 Rotate pillow sleeve
assembly forward or
backward so the nasal
pillows sit comfortably at
the entrance of the nostrils.



4 Connect one end of the
main air tubing to the
flow generator and the
other end to the swivel.
Turn flow generator on.



cap

5 The tubing may be
positioned on either the left
or right hand side. To change
the tube position, swap the
tube on one side with the
cap on the other side.



6 You may choose to
position the tube over the
head (secured by the tube
retainer if preferred).

Note: For complete fitting instructions, refer to the
Mirage Swift Nasal Pillows System User's Guide.

Specifications may change without notice. 60861/1 04 04

MIRAGE
SWIFT™
NASAL PILLOWS SYSTEM