


PILOT 12/24 CONTROL PANEL GUIDE

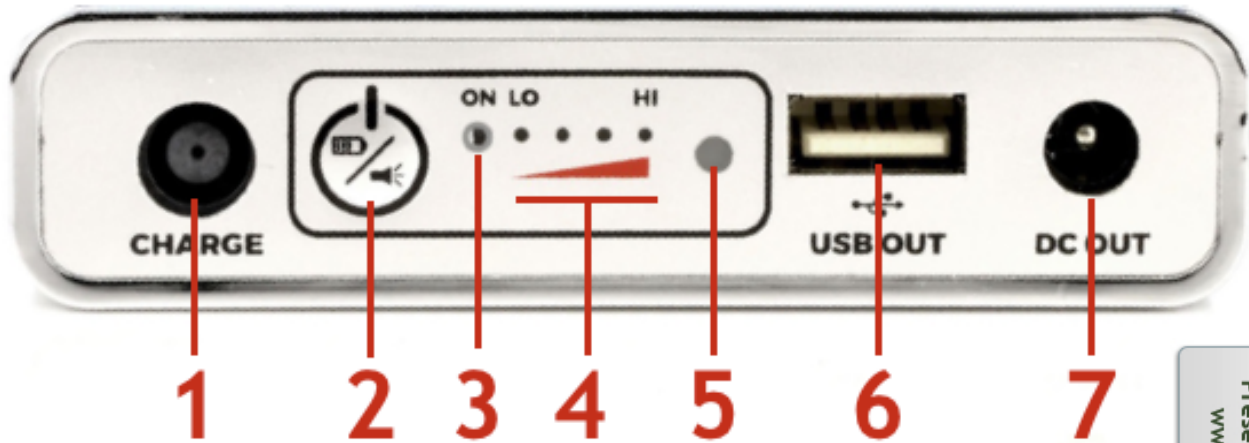

1. AC Charge Port

- Use the PAP machine's AC power cord to charge battery
- Leave AC Power Cord connected for use as a Backup Power Source.

2. Multi Function Button

- TAP button to display charge level indicator (GREEN LED's)
- HOLD button to activate / deactivate LED FLASHLIGHT

3. Blue "ON" Indicator Light (LED)

- The BLUE LED will activate when connected and sending power to a device.

4. Green Charge Level Indicator Lights (LED)

4 Solid Green Lights	$\geq 90\%$ charge remaining
3 Solid Green Lights	65% - 90% charge remaining
2 Solid Green Lights	40% - 65% charge remaining
1 Solid Green Light	10% - 40% charge remaining
4 Flashing Green Lights	Battery is CHARGING

5. Flashlight (LED)

6. USB Port (5V, 2.5A Out)

- Used for charging handheld devices.
- Once a USB cable is connected, tap the "Multi Function Button" (no. 2) to activate battery.

7. DC Out Port

- This port is used to connect a compatible PAP device to the battery

Presented With Compliments From
www.DirectHomeMedical.com

