







SoClean Adapter for Fisher & Paykel ICON™

The adapter for the Fisher & Paykel ICON™ is a special fitting. The adapter replaces the original elbow fitting on your machine. If you need help connecting your adapter, contact support at SoClean. (See reverse for contact information.)

Connecting the Adapter



1. Detach the injection fitting and short hose from the main SoClean hose.
2. Disconnect the short hose from the inside of the injection fitting.
3. Install the short hose to the stem on the inside of the new ICON adapter.
4. Connect the main hose from the SoClean to the stem on the outside of the ICON adapter.
5. Detach your CPAP hose from the elbow fitting on the back of your ICON CPAP.
6. Detach the original elbow fitting from the back of the ICON CPAP.

(Continued on next page)

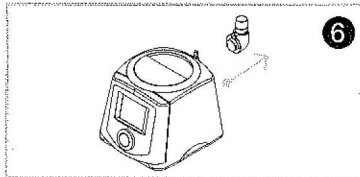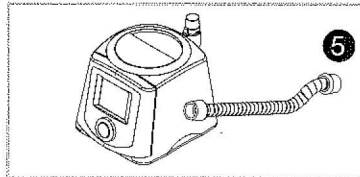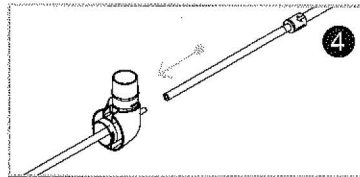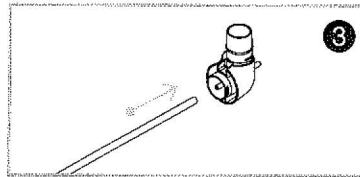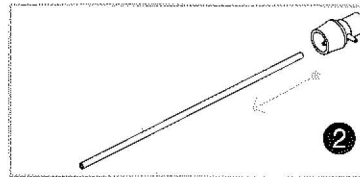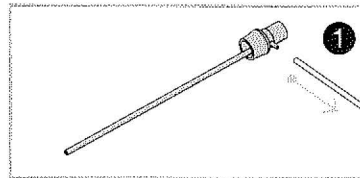





SoClean Adapter for Fisher & Paykel ICON™ ... Continued

1. Grasping the ICON adapter, guide the short black hose into the humidifier reservoir using a pushing and twisting motion. (Note: It may be necessary to trim off some of the hose. The end should rest near but not touch the bottom of the reservoir.) Snap the adapter into place.
2. Connect your CPAP hose to the new ICON adapter. You are now ready to use your SoClean.

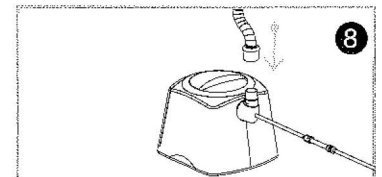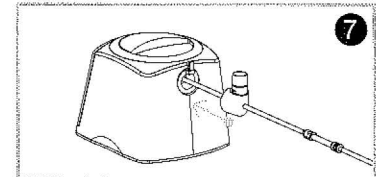

Disconnecting the Adapter

1. Disconnect your CPAP hose from the ICON heated hose adapter.
2. Remove the ICON adapter from your CPAP unit.
3. Install your original elbow fitting to your CPAP unit.
4. Connect your CPAP hose to the elbow fitting. You are now ready to use your CPAP unit.

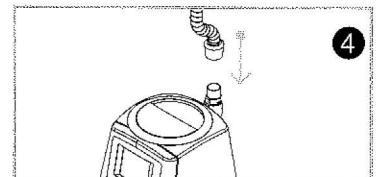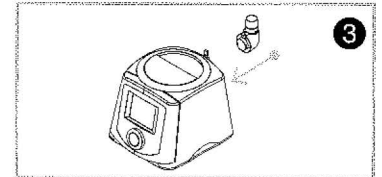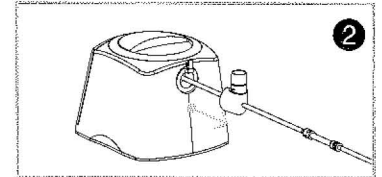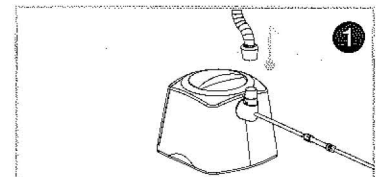

[CLICK TO LEARN MORE](#)