



Care Instructions


- Daily Cleaning
- On the Go Deep Cleaning
- Optional Disinfection



[CLICK TO LEARN MORE](#)


Daily Cleaning Instructions:

For full care instructions, see the Flyp Owner's Manual or Quick Start Guide.

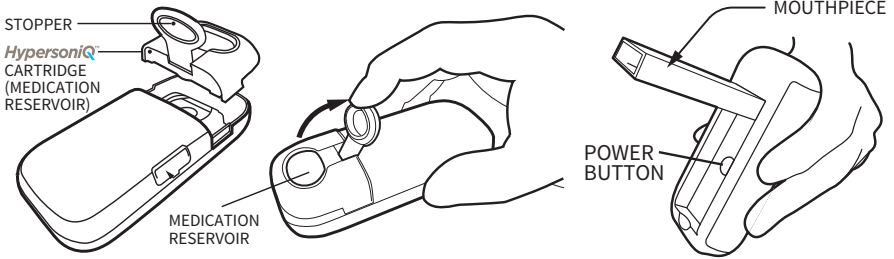

STEP 1


OPEN THE RESERVOIR STOPPER

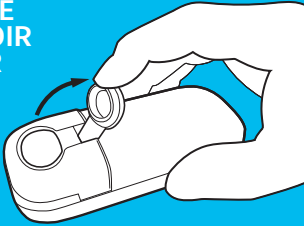

Check the Medication Reservoir to make sure it is empty.

STEP 2


REMOVE AND CLEAN

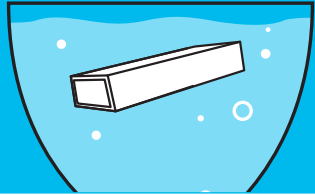

Rinse reservoir. Clean mouthpiece with mild soap & water.

STEP 3


FILL WITH WATER

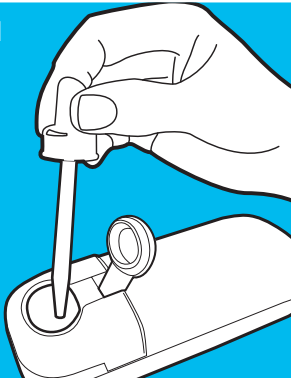

Reinsert the Reservoir and fill with water

STEP 4

RUN FOR 1-2 MINUTES.

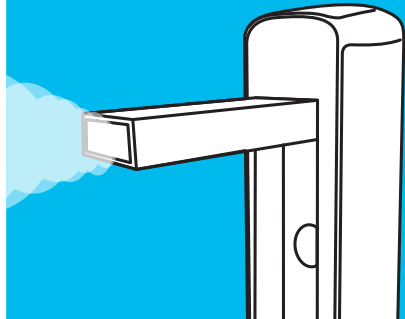

Press and hold power button for 5 seconds. Run for 1-2 minutes or until empty.

Deep Cleaning Instructions:

PART A



Note: Ampules may appear different than above

1. Break off one ampule of “A” Cleaning Solution and one ampule of “B” Distilled Water from your package. Set both aside.
2. Open the stopper on the HypersonIQ® cartridge (medication reservoir).
3. Twist off the top of one “A” Cleaning Solution ampule.
4. Insert the “A” Cleaning Solution ampule into reservoir and squeeze to empty contents.
5. Close the reservoir, hold Flyp vertically, and gently raise the Mouthpiece with your index finger.
6. Press and hold the power button for 5 seconds. Blue AND yellow indicator lights will start to blink and Flyp will produce a pulsing aerosol.
7. Allow the aerosol mist to flow from the raised Mouthpiece until cleaning solution is completely gone.

PART B



8. Power Flyp off by pressing the power button.
9. Remove HypersonIQ cartridge and shake out any excess cleaning solution.
10. Replace the HypersonIQ cartridge.
11. Open the stopper on the HypersonIQ cartridge (medication reservoir).
12. Twist off the top of the “B” Distilled Water ampule.
13. Insert the “B” Distilled Water ampule into reservoir and squeeze to empty contents.
14. Repeat steps 5 - 9.
15. Let the main unit, cartridge, and mouthpiece dry on a clean, dry towel.
16. Allow the interior to dry fully prior to using again.

Optional Disinfection Instructions:

HYPERSONIQ CARTRIDGE AND MOUTHPIECE

STEP 1

Remove the HypersonIQ Cartridge and mouthpiece from Flyp.

STEP 2

Soak the mouthpiece and cartridge in 70% ethyl alcohol for 10 minutes. Make sure both are completely submerged in the alcohol.

STEP 3

Remove mouthpiece and cartridge from the alcohol bath and rinse under running water.

STEP 4

Let dry on a clean, dry towel.



EXTERIOR

STEP 1

Wipe exterior with clean, dry towel moistened with ethyl alcohol.

STEP 2

Allow Flyp to sit for 5 minutes.

STEP 3

Moisten with a clean dry towel with water and wipe exterior to remove the ethyl alcohol.

STEP 4

Let dry on a clean, dry towel.



IMPORTANT USE ONLY AS DIRECTED. The purpose of these cleaning instructions is to ensure a cleaner and more efficient operation of your nebulizer. It is important to use the cleaning solution (Vial #1) FOLLOWED by the distilled water provided (Vial # 2) in this package before using your nebulizer. Before you use the On-the-Go Cleaning vials or your nebulizer, consult with your healthcare provider about all of your current medical conditions and allergies. Be sure to tell your doctor if you are pregnant, think you might be pregnant, or plan to become pregnant. Tell your healthcare provider if you are breast-feeding. Tell your healthcare provider about all other medications you are currently using. Tell your caregiver right away or call 911 if you have chest pain, trouble breathing, swelling in your hands or feet, tiredness, confusion, uneven heart rate, extreme thirst, increased or decreased urination, leg discomfort, muscle weakness or limp feeling, or if you feel like you might pass out. **ALLERGY WARNING: THIS PRODUCT CONTAINS 5% ACETIC ACID (VINEGAR).**