


flyp TIPS for best results

Tilt Down & Inhale

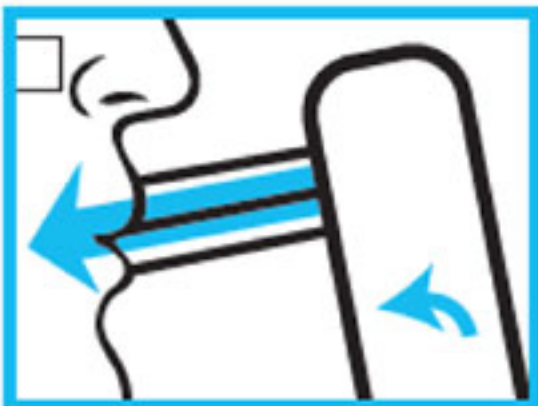

Exhale Off To Side




NOT In Device



Tilt the device towards you so the mouthpiece is slightly downward. Inhale medication normally. Exhale to the side of the mouthpiece. *Avoid exhaling back into FLYP device.*

Charge

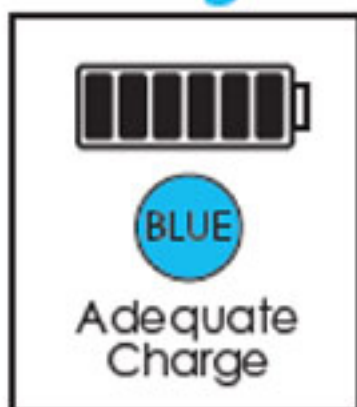


Make sure Flyp is charged before using.

Tap



If an air bubble occurs in the reservoir, it may disrupt performance. Simply tap the device to eliminate the air bubble.

Dab

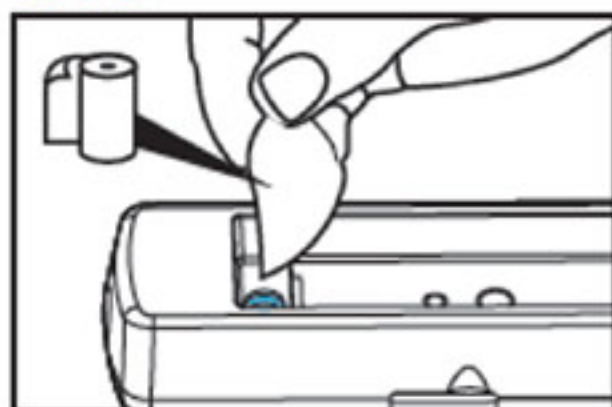

If medication builds up on the front of the disk, just remove the mouthpiece and gently dab the medication with a paper towel.