
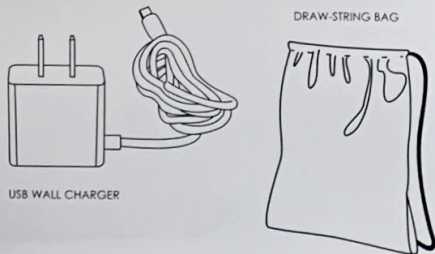
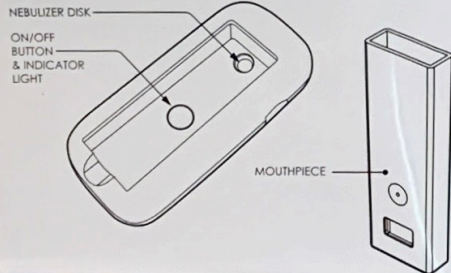


1 LOOK

STEP 1


Learn the parts.

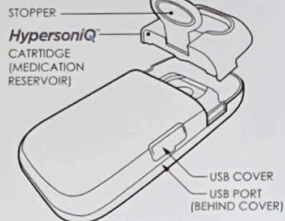


2 CHARGE

STEP 1

Connect USB Wall Charger to USB Port and plug into wall outlet. Flyp will fully charge in 1.5 to 2 hours.

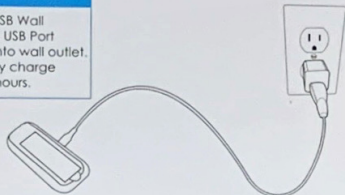

Indicator lights when charging

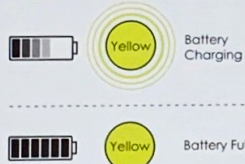


3 FILL

STEP 1


Open the Reservoir.

Do not remove Medication Reservoir from the nebulizer.

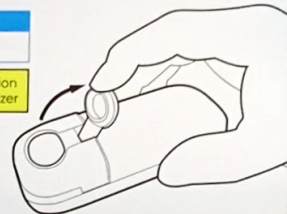

STEP 2

Insert the medication ampule into the reservoir.

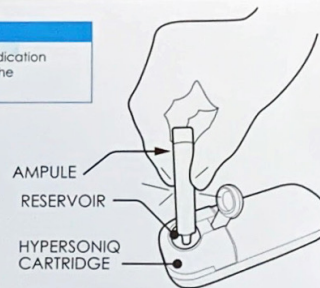

STEP 3

Close the Reservoir.




4 ON/OFF

STEP 1

Flip up the Mouthpiece.


Make sure unit is unplugged from charger.

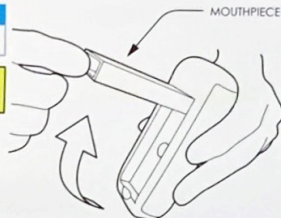


STEP 2

Press the ON/OFF button once to turn the unit on.

Flyp should NEVER BE used while charging. The USB Wall Charger should be removed before using, cleaning or disinfecting.

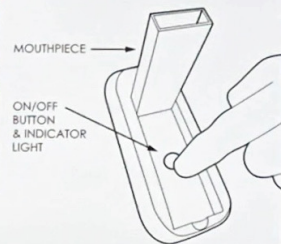

Indicator lights when operating

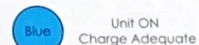


CLICK TO LEARN MORE

5 INHALE

STEP 1

1. Place the Mouthpiece in your mouth.
2. Inhale the medication.
3. Unit turns off automatically at 10 minutes.

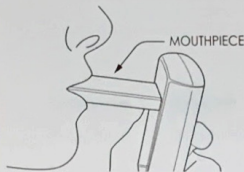

6 CLEAN & DRY

IMPORTANT: CLEAN AFTER EVERY USE

After each use, it is extremely important to clear away medication residue from the inside of the Flyp unit, the Medication Reservoir, and the Mouthpiece. Buildup of medication will damage the unit and inhibit performance. READ USER GUIDE FOR FULL INSTRUCTIONS. Visit flypnebulizer.com for videos and additional information to help you care for your FLYP.

OPEN THE RESERVOIR STOPPER

STEP 1

Check the Medication Reservoir to make sure it is empty.

REMOVE AND CLEAN

STEP 2

Clean Reservoir and Mouthpiece with mild soap & water.

FILL WITH WATER

STEP 3

Reinsert the Reservoir and fill with water.

RUN FOR A MINIMUM OF 60 SECONDS

STEP 4

Operate the unit for 60 seconds while holding upright.

DEEP CLEANING

Residual medication can accumulate over time and cause buildup on the disk which may slow aerosol production. To prevent this, a deep cleaning with distilled white vinegar (or Flyp cleaning solution) on a weekly or as-needed basis is recommended.

NEBULIZER DISK

STEP 1:

Check the reservoir and make sure it is empty.

STEP 2:

Fill the reservoir approximately half-way with distilled white vinegar (or Flyp cleaning solution).


STEP 3: Self clearing function:

To activate this mode, press and hold power button for 3 seconds to enter "clearing mode". Operate the unit for 2-3 minutes or until the reservoir is empty.

STEP 4: Power off Flyp.

STEP 5: Remove reservoir and mouthpiece. Rinse both under running water and let dry.

OPTIONAL DISINFECTION

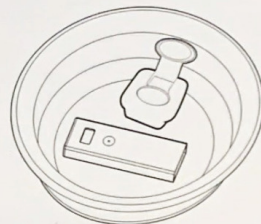

HYPERSONIQ CARTRIDGE AND MOUTHPIECE

STEP 1

Remove the HypersoniQ Cartridge and mouthpiece from Flyp.

STEP 2

Soak the mouthpiece and cartridge in 70% ethyl alcohol for 10 minutes. Make sure both are completely submerged in the alcohol.

STEP 3

Remove mouthpiece and cartridge from the alcohol bath and rinse under running water.

STEP 4

Let dry on a clean, dry towel.

EXTERIOR

STEP 1

Wipe exterior with clean, dry towel moistened with ethyl alcohol.

STEP 2

Allow Flyp to sit for 5 minutes.

STEP 3

Moisten with a clean dry towel with water and wipe exterior to remove the ethyl alcohol.

STEP 4

Let dry on a clean, dry towel.



**CONVEXITY
SCIENTIFIC**

418 Meadow Street
Fairfield, CT, 06824

www.flypnebulizer.com
www.convexityscientific.com

flyp

Portable Vibrating
Mesh Nebulizer
Quick Setup Guide

Start
Here

Model
fn2000m

✓ Fill
✓ Click
✓ Breathe

Register Your Flyp

Visit www.flypnebulizer.com to register your purchase or to order replacement items including HypersoniQ cartridges



HypersoniQ™
Replacement Cartridges

Note: HypersoniQ™ Cartridge should be replaced as needed, or every 3 months.

