

[CLICK TO LEARN MORE](#)

How to use your Philips InnoSpire Go

The following is a basic guide for using a Philips InnoSpire Go portable mesh nebulizer. You should always refer to the instructions for use supplied for more detailed and specific instruction.



1

Attach the mouthpiece to the handset.



2

Lift the lid of the green medication chamber and empty the contents of your medication vial into the chamber.



3

Make sure you don't exceed the maximum fill volume, which is marked by the word MAX under the hinge.



4

Close the lid.



5

If you are using a mask, you should attach the facemask adapter to the mouthpiece and then attach your mask to the adapter.



6

If you are going to use the power adapter, plug the cable into the socket on the handset and plug the adapter into the power outlet.

How to use your Philips InnoSpire Go

Continued



Turn on your nebulizer by pressing the ON/OFF button on the handset.



If the light is green, the battery is charged and ready to go.



If the light is amber, there's enough charge for at least one more treatment, but be sure to charge your nebulizer after your treatment.



If the light flashes amber and then switches off, there's not enough charge to take a treatment. If this is the case, simply use the power adapter to continue.



Look to make sure that aerosol is coming out of the mouthpiece or the facemask.



If using the mouthpiece, hold the handset in your hand, place the mouthpiece between your teeth and seal your lips around it. Breathe normally through your mouth.

How to use your Philips InnoSpire Go

Continued



13

If using a facemask, hold the handset in your hand, gently press the mask against your face and breathe normally through your mouth.



14

Do not tilt the handset more than 45° in any direction as this might stop it from completely nebulizing the medication in the chamber.



15

If you need a rest, just press the ON/OFF button to pause your treatment. When you're ready to continue, press the ON/OFF button again.



16

Your treatment will take about four minutes and is finished when the nebulizer beeps and the light flashes.



17

The device will then turn itself off automatically.

Cleaning your Philips InnoSpire Go

With just two pieces, cleaning and maintaining the InnoSpire Go portable mesh nebulizer is simple and easy. It is very important to follow the cleaning instructions, otherwise the mesh inside the mouthpiece may become clogged with dried medication.

After each use



1 Pour away any leftover medication from the medication chamber.



2 Press the release button to separate the mouthpiece from the handset.



3 Rinse the mouthpiece thoroughly under running water.



4 Shake off excess water, place it on its side and leave to air-dry fully before storing.

Daily cleaning



1 Wash the mouthpiece by hand in warm, soapy water using regular dishwashing liquid.



2 Rinse the mouthpiece thoroughly under running water.

Cleaning your Philips InnoSpire Go



Shake off excess water, place it on its side and leave to air-dry fully before storing.

Weekly disinfection



Boil the mouthpiece in water for 10 minutes with the medication chamber lid open and enough water in the pan to stop it resting on the bottom or boiling dry.



Shake off excess water, place it on its side and leave to air-dry fully before storing.

General care



Cleaning the handset couldn't be easier, just wipe with a clean damp cloth weekly or whenever needed.



The mask, if used, should be washed once a week in warm soapy water for 2 minutes or leave to soak for 10 minutes. Rinse it under running water and leave to air-dry.



When you're not using your InnoSpire Go, store it in its original case to protect.