


# FILTER SET WITH REMOVAL TOOL



PART# 085F0012

This Filter Set includes a replacement filter and filter removal tool for the PARI Vios® PRO, PRONEB® Ultra, and SinuStar™ compressors. Under normal conditions, the filter on the front panel should be replaced after 500 hours of use or sooner if it appears soiled (grey), clogged, or wet. Check the filter every 10-12 treatments.

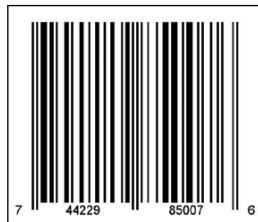

## INSTRUCTIONS FOR FILTER REPLACEMENT

Push the filter tool firmly over the filter and rotate the filter removal tool counter clockwise a full 8 turns until loose and pull the filter out (Figs. A and B). Discard the dirty filter and tool. The filter tool prevents unwanted removal of the filter or a lost filter.

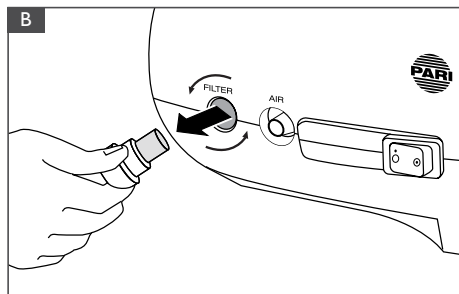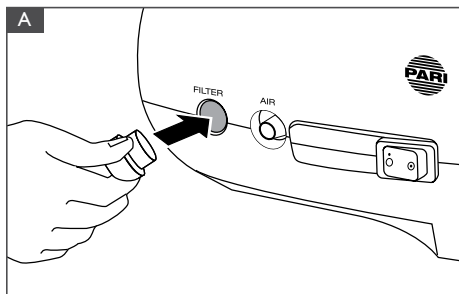

Once the filter is removed, tilt the compressor so that the face of the unit is facing the floor. Gently tap on the back of the compressor to remove any unwanted materials from the filter area. Place the compressor on a stable surface and using clean hands, push a new filter into the filter receptacle. Dispose of used filter and filter removal tool after use.

**NOTE:** DO NOT ATTEMPT TO WASH AND REUSE AIR FILTERS. DO NOT USE SUBSTITUTE FILTER MATERIAL COMPRESSOR DAMAGE MAY RESULT.

### **WARNING**

This device contains small parts that might be a choking hazard if swallowed. Please keep all small parts out of the reach of children

Manufactured for:

**PARI Respiratory Equipment, Inc.**

2412 PARI Way, Midlothian, VA 23112 USA

Tel: 800.FAST.NEB (327.8632) and 804.253.7274

Fax: 800.727.4112 | Product.Info@PARI.com

Online at: [www.PARI.com](http://www.PARI.com)



© 2015 PARI Respiratory Equipment, Inc.  
085D0012 Rev D 11-15