The Transcend 365 Portable Water Filter Kit filters tap water for use with your Transcend 365 miniCPAP™ device. This portable water filtration system is perfect for travel or when distilled water isn't readily accessible to you.

Note that failure to use distilled water will damage the heater in the humidifier and may require repair or replacement. Alternatively, you may use the Transcend 365 Portable Water Filter Kit to filter water for use with your Transcend 365 miniCPAP™ device.

Using your Transcend 365 Portable Water Filter Kit:

1. Pull the filter sleeve off of the base. Note that this piece does not twist off, it must be pulled from the base with force.

2. Unscrew the cap from the filter and store it for safekeeping.

3. Align the base and filter with the reservoir's opening by connecting the magnets on each piece. The base should securely snap into place.

4. Clean the threads on the filter and filter sleeve before screwing them together to create a tight seal.


5. Fill the funnel only to the fill line marked on the filter sleeve to avoid overflowing your reservoir.

6. Once you've finished filling the reservoir to its max fill line, unscrew the filter sleeve from the filter and replace the filter cap. The filter sleeve snaps back onto the base for storage.

Note: Do not continue to use your filter once it indicates it's time to replace it, as this may lead to insufficient water filtration and can damage the heater in your humidifier. The ZeroWater™ Portable Travel Bottle Replacement filters can be purchased from your Transcend dealer or most major retailers.

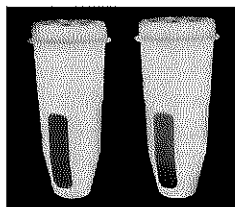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