

**ResMed**

**AirFit™ F30i**

Full face mask


Actual Size: 9 3/4" (Width) x 4" (Height)

Ruler is for scaling purposes.



How to use the fitting template:


1. Please ensure the template is printed to scale
2. Cut out along dotted line
3. Refer to diagram on template to determine appropriate cushion size
4. Refer to diagram on right to determine appropriate frame size

[CLICK TO LEARN MORE](#)

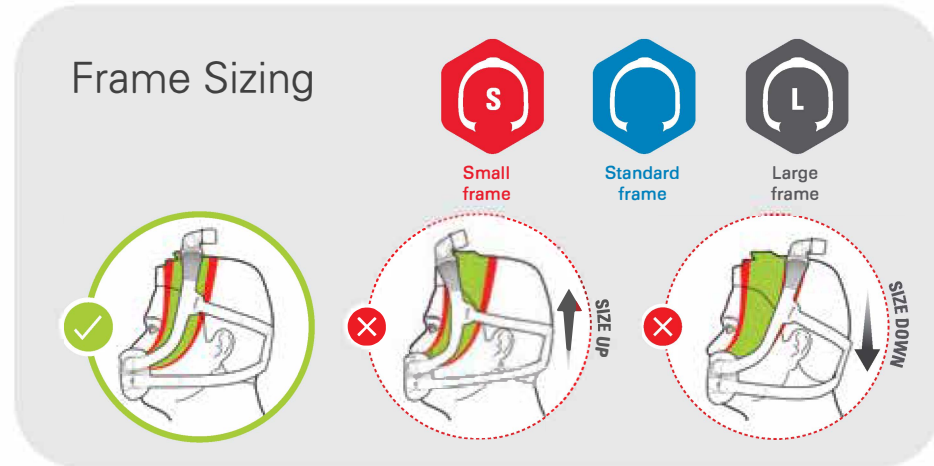

Cushion Sizing

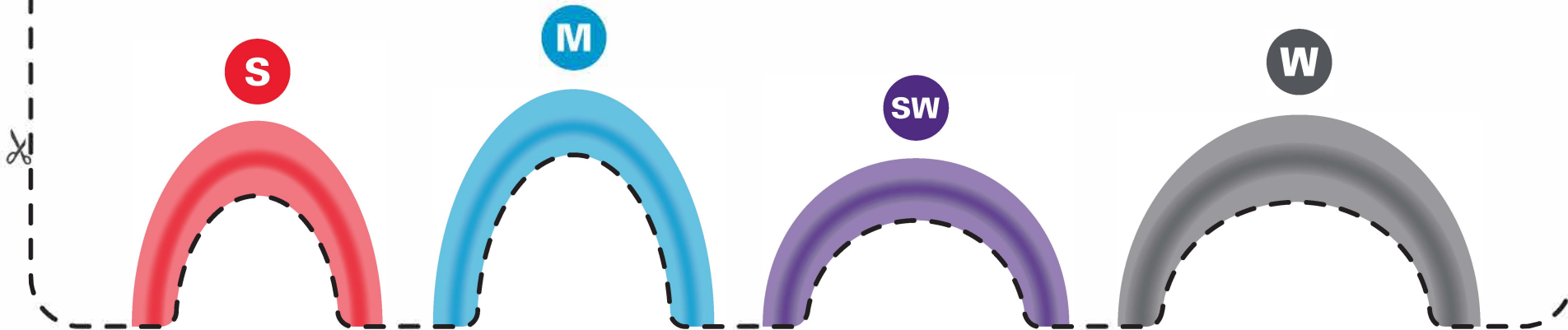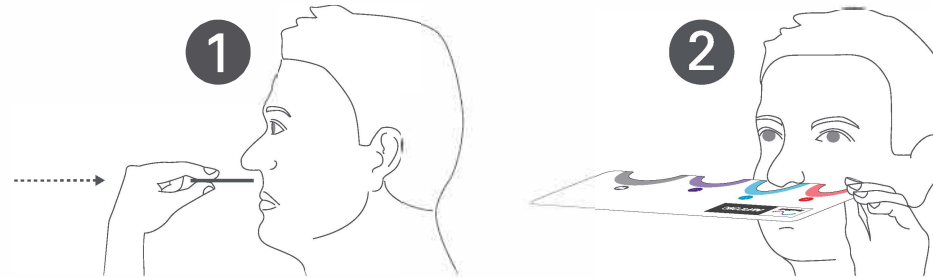



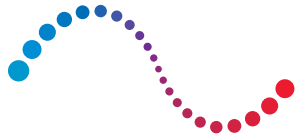





**ResMed**

**AirFit™ F30i**

Full face mask
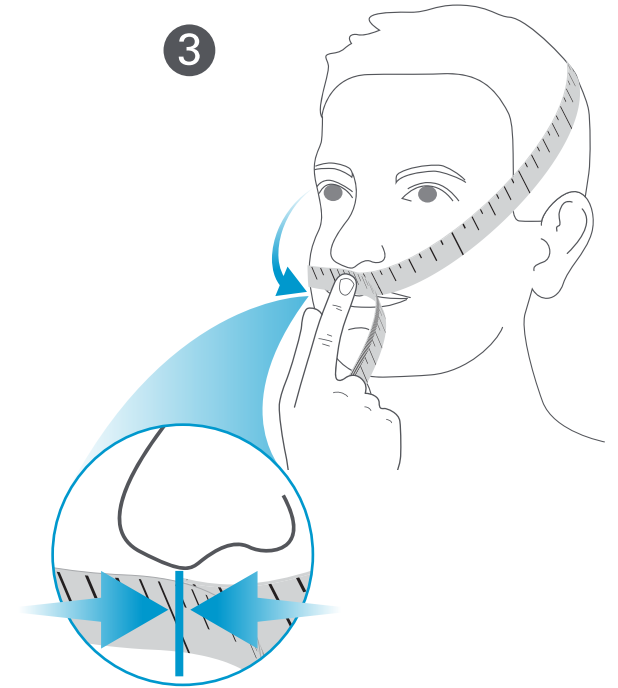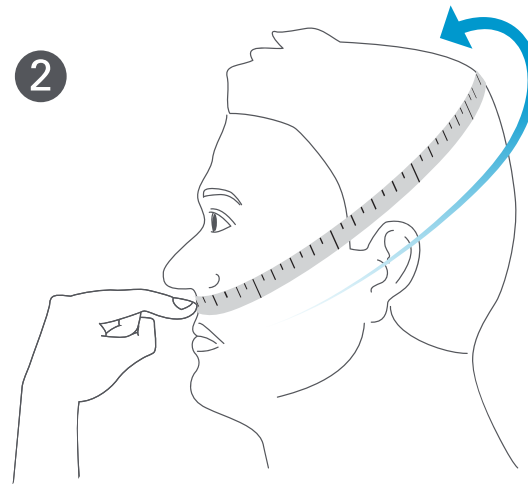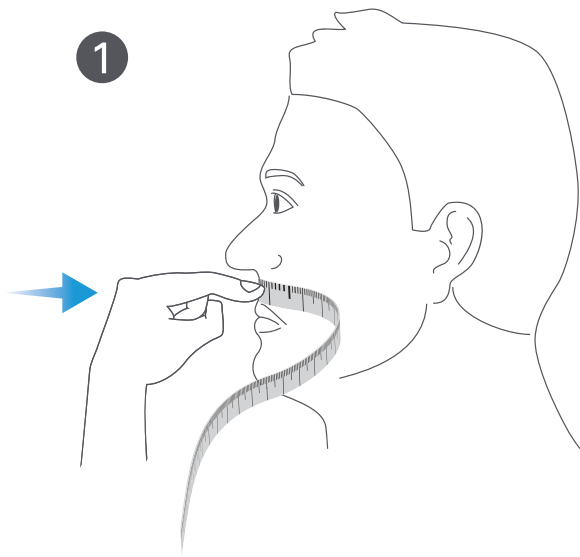
## Frame Fitting Instructions



 Small frame	 Standard frame	 Large frame
<b>575-618mm</b>	<b>619-653mm</b>	<b>654-695mm</b>
<b>22<sup>5</sup>/<sub>8</sub>" – 24<sup>3</sup>/<sub>8</sub>"</b>	<b>24<sup>3</sup>/<sub>8</sub>" – 25<sup>11</sup>/<sub>16</sub>"</b>	<b>25<sup>11</sup>/<sub>16</sub>" – 27<sup>3</sup>/<sub>8</sub>"</b>