



**DirectHome
MEDICAL**
[CLICK TO LEARN MORE](#)

DreamPort™ Application Instructions




1

DreamPort™ Application Instructions

Step 1

The first step to applying your DreamPorts™ is to thoroughly clean your nose area.

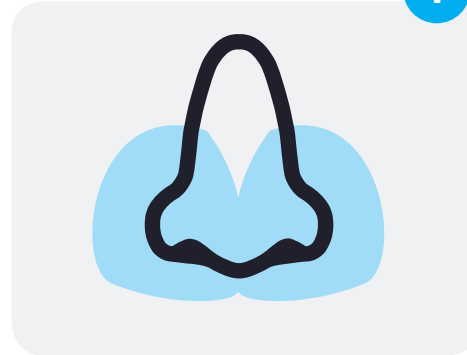

Clean areas around nostrils of makeup, oil or dirt with DreamPrep™ wipes or other mild astringent like alcohol or witch hazel wipes. Then allow your nose to air dry or pat dry with a towel.



2

Step 2

Apply the DreamPort™ adhesives by applying pressure around the nostril - ensuring you have a good seal. Repeat the process to apply the second DreamPort™.



3

Step 3

Click the applied DreamPorts™ into the DreamWay™ connector, then attach to your CPAP unit.



4

Step 4

Settle in for the best bleeping night's sleep you've ever had!



DreamPort™ Instrucciones de aplicación

Visite www.bleepsleep.com/instructions para un video con instrucciones detalladas.




1

DreamPort™ Instrucciones de aplicación

Etapa 1

Limpié bien su nariz y el área alrededor de su nariz.

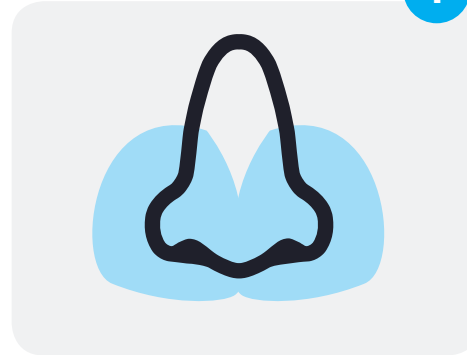

Limpié las fosas nasales de maquillaje, aceite o suciedad con la toallita DreamPrep™. Después, deja que la nariz se seque al aire o seque con una toalla.



2

Etapa 2

Aplicar el adhesivo DreamPort™ por aplicando presión sobre las fosas nasales - asegúrese que tiene una buena conexión.



3

Etapa 3

Conecta los DreamPorts™ a la interfaz DreamWay™ con un clic audible, entonces conecta a su dispositivo para CPAP.



4

Etapa 4

Prepárese para dormir bien!

