Presented With Compliments From
www.DirectHomeMedical.com

HYBRID™

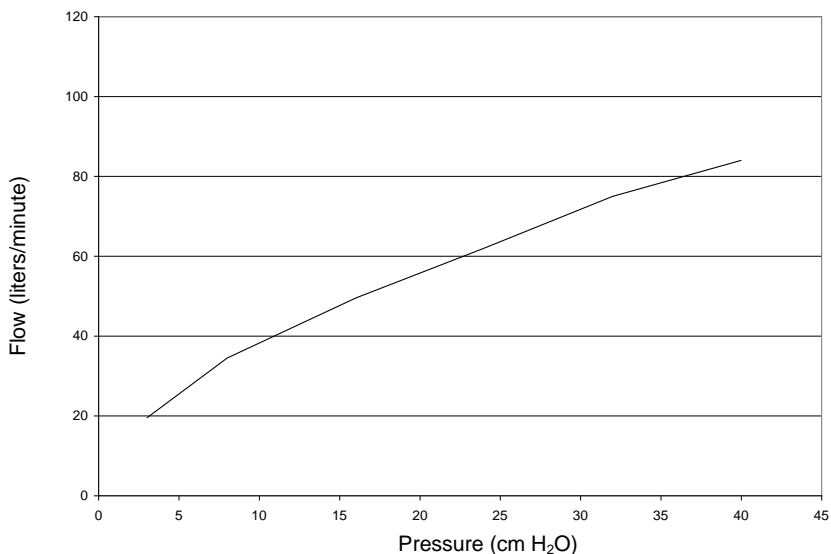
Hybrid™ Interface

Instructions for Use – 05080 Rev E

EN: ENGLISH

LOT _____

Passive Exhalation Port Flow

The deadspace associated with this mask is less than 125 cm³.

LATEX FREE

Patents Pending

Intended Use

The Hybrid is intended for use by adults (> 30 kg) as a patient interface for CPAP or Bi-Level positive-pressure ventilation devices for treatment of respiratory insufficiencies and obstructive sleep apnea.

For homecare applications, the Hybrid may be reused multiple times by a single patient. For institutional applications (i.e. sleep lab, hospital, or other clinical settings), this interface may be reused multiple times by multiple patients.

Cautions

- RX Only
- Storage Temperature: -20° to 60°C, Operational Temperature: 5° to 40°C, Humidity: 0–95%

Warnings

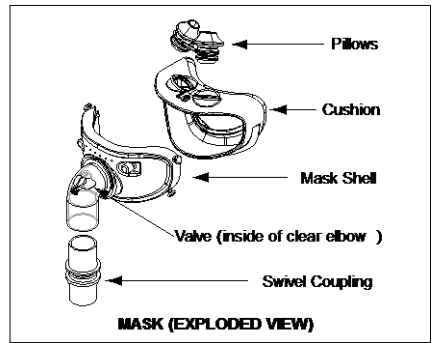
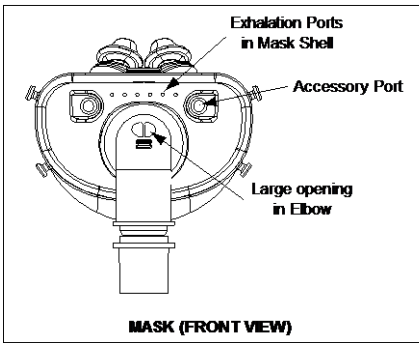
- This mask should be used only with CPAP or Bi-level systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-level system is turned on and operating properly. The exhalation ports should never be blocked.
Explanation of the Warning: CPAP/Bi-level systems are intended to be used with special masks which have vent holes or exhalation ports to allow continuous flow of air out of the mask. When the CPAP/Bi-level machine is turned on and functioning properly, new air from the machine flushes the exhaled air out through the attached mask exhalation ports. However, when the CPAP/Bi-level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation. This warning applies to most models of CPAP and Bi-level systems.
- At low CPAP pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- A minimum pressure of 3.0 cm H₂O at the mask is required.
- The anti-asphyxia valve allows the patient to breathe room air in the case that the CPAP/Bi-level device connected to the mask is not operating or is turned “off”. The valve closes at operating pressures of 3.0 cm H₂O and above. The mask should not be used if the anti-asphyxia valve is missing, damaged, or not working properly. The large opening at the valve should never be blocked.
- Do not use the mask if vomiting or feeling nauseous. Failure to arouse and remove the mask after vomiting could result in aspiration of vomitus. To minimize the risk of vomiting during sleep, avoid eating or drinking for three hours prior to sleep. This mask is not recommended if the patient is taking a prescription drug that may cause vomiting.
- Due to the risk of transmitting infection, disinfect between patients when reusing in the institutional setting.
- The mask may deactivate the pressure alarm on some ventilation systems. Confirm that your ventilation system pressure alarm works with the product before using the pressure alarm.
- If you notice redness or irritation of your skin, contact your physician. The cushions and pillows consist of silicone. Other mask components consist of silicone, polycarbonate, polypropylene, nylon, polyester, and neoprene.
- Any unusual chest discomfort, shortness of breath, stomach distention, belching or severe headache during or immediately after use should be immediately reported to your physician.
- If oxygen is used with the CPAP or Bi-level device, the oxygen flow must be turned off when the flow generator is not operating.
Explanation of the warning: When the CPAP or Bi-level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the flow generator enclosure. Oxygen accumulated in the CPAP or Bi-level device enclosure will create a risk of fire. This warning applies to most types of CPAP or Bi-level flow generators. Do not smoke while oxygen is in use.
Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.

Contraindications

This mask may not be suitable for persons with the following conditions: impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia. It should not be used if patient is uncooperative, obtunded, unresponsive, or unable to remove the mask.



Before Each Use

1. Clean the mask as described in the cleaning instructions. For single patient use in the home, see Cleaning Instructions – Homecare Applications. For multiple patient reuse in institutional settings, see Cleaning and Disinfecting Instructions – Institutional Applications.
2. Inspect the mask for deterioration as described in the cleaning instructions.
3. Inspect the anti-asphyxia valve located within the clear elbow. With the CPAP or Bi-level system turned off, verify that the valve is positioned so that room air can flow through the large opening near the valve in the elbow. Turn on the system to 3 cmH₂O; the valve should close the large opening and system air should flow into the mask. If the valve does not close or does not function properly, replace the mask.

Assembling the Mask

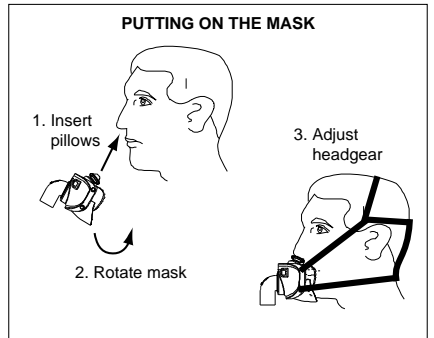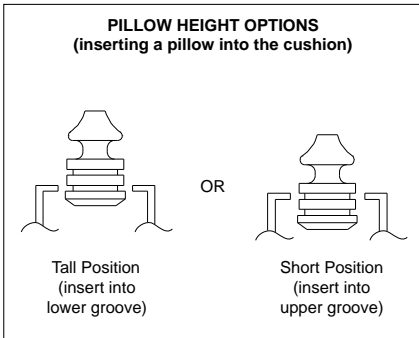1. Attach the swivel coupling to the mask elbow.
2. Insert nasal pillows into cushion. NOTE: Pillows can be attached by their lower groove (Tallest Height Position) or their upper groove (Shortest Height Position) in order to optimize your seal.
3. Attach the cushion to the mask shell.
4. Attach headgear by snapping the buckles of the headgear onto the posts on the mask shell.
5. Attach the circuit tube from the CPAP or Bi-level ventilation system to the swivel coupling.

Putting on the Mask

1. Clean and thoroughly dry your face before putting on the mask.
2. Insert the pillows into your nose and then rotate the mask down over your mouth. NOTE: The upper wall of the cushion should sit above your mouth on your upper lip.
3. Pull the headgear back over the top of your head.
4. Turn on CPAP or Bi-level system and make sure it is functioning properly. Connect the mask.

Adjusting the Mask

1. Unhook the hook-and-loop closures of each of the five headgear straps individually and gently pull until the mask sits squarely on your face. The mask should fit snug enough to prevent large leaks yet still be comfortable. In general, a looser fit is better. Over-tightening may actually make leaks worse.
2. Breathe normally, and continue making minor adjustments until you have a comfortable fit with minimal leaks. If you are unable to achieve a comfortable fit or to resolve any leaks around the mouth, you may need an alternative cushion size. If you are unable to achieve a comfortable fit or to resolve any leaks at the nose, you may need an alternative pillow size.
3. To remove the mask, you do not need to unhook each strap. Just unsnap the buckles of the headgear from the mask, and then lift the mask off over your head.

Cleaning Instructions – Homecare Applications

After Each Use:

1. Remove the headgear from the mask; it will be washed separately.
2. Disassemble swivel coupling and cushion from the mask. Remove pillows from the cushion.

3. Wash the swivel coupling, cushion, pillows, and the remaining mask in warm water with a mild dishwashing detergent. Do not use alcohol or petroleum products or strong cleaners.
4. Make sure that the exhalation ports are free of any obstructions. Use a toothpick or similar item to remove any debris.
5. Rinse thoroughly. Allow all components to completely air-dry, away from direct sunlight.
6. Carefully inspect each component for any wear or damage. If you see any cracks or crazing in the plastic, or any tears or hardening of the silicone components, do not use the mask. Discoloration is normal after several washings and should not be considered sign of wear. Contact your healthcare provider for a replacement.

At Least Every 7 Days:

1. Hand wash the headgear in warm water with a mild laundry detergent. Do not use bleach.
2. Rinse thoroughly. Allow to completely air-dry, away from direct sunlight.
3. Carefully inspect the headgear for wear or damage. Do not use if you notice significant tears or fraying. Contact your healthcare provider for a replacement.
4. After repeated washings, the headgear may stretch out enough to require re-adjustment.

Cleaning and Disinfecting Instructions – Institutional Applications

Warning: Home users should not attempt to sterilize or use disinfectants on the device components.

After Each Use:

1. Disassemble headgear, swivel, and cushion from mask. Remove pillows from cushion.
2. Wash the swivel coupling, cushion, pillows, and the remaining mask in warm water with a mild dishwashing detergent. Do not use alcohol or petroleum products or strong cleaners.
3. Make sure that the exhalation ports are free of any obstructions. Use a toothpick or similar item to remove any debris.
4. Rinse thoroughly (3) three times.
5. If any residue remains on any component after cleaning, discard entire assembly and replace.
6. Prepare a solution of Cidex OPA (*ortho*-Phthaldehyde solution) per manufacturer's instructions. Verify strength of Cidex OPA solution using Cidex OPA test strips.
7. Submerge all components into the prepared Cidex OPA solution for twelve (12) minutes.
8. Remove components. Shake excess solution from interior surfaces of components.
9. Rinse assembly three (3) times with clear water.
10. Shake excess water from interior surfaces of components.
11. Allow all components to completely air-dry, away from direct sunlight.
12. Carefully inspect each component for any wear or damage. If you see any cracks or crazing in the plastic, or any tears or hardening of the silicone components, do not use the mask. Discoloration is normal after several washings and should not be considered sign of wear. Contact your healthcare provider for a replacement.

Manufactured
for:

RESPCARE

Coconut Creek, Florida 33073 USA
(866)-216-7337 +1(561)-208-3770

Manufactured
by:

GaleMed Corp.
39, Sec. 3, Haijing E. Road
Xiamen, Fujian Province, China
Made in China

Obelis S.A.
Av. De Tervueren 34, Bte 44
1040 Brussels, Belgium

Manufactured for RespCare Inc. Hybrid is a trademark of RespCare, Inc. *Covered by Patent Numbers D557801, D550352, D550836, and Patents Pending*

Technical Support: (800)-200-9842 + 1 (561)-208-3770 International
Website: WWW.HYBRIDMASK.COM

Reorder Information

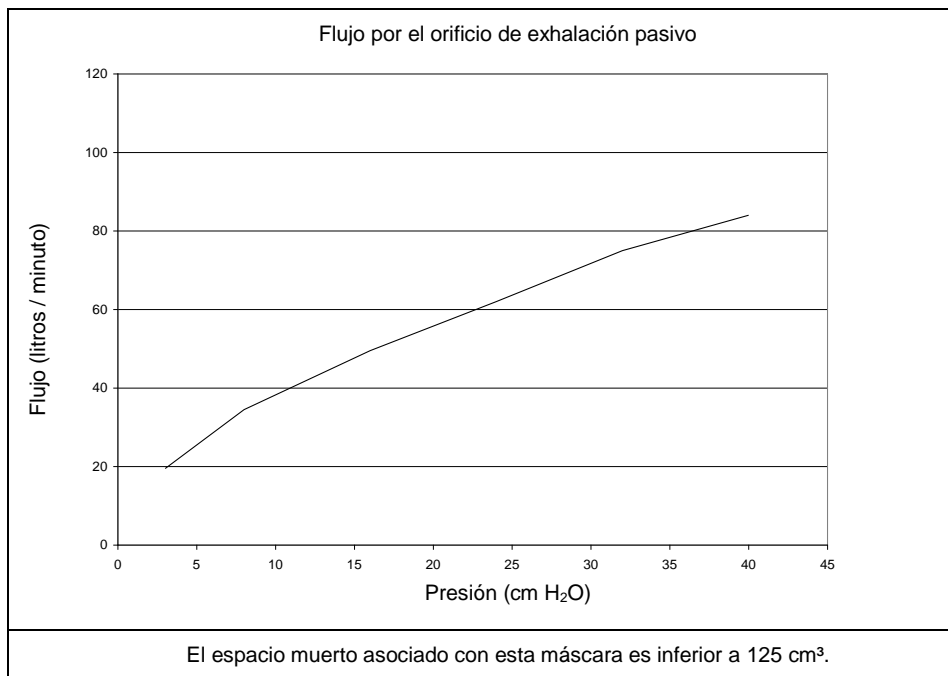
Part Number	Description	Part Number	Description
HYB500	Hybrid Interface – Complete System	HYB521	Pillows, <i>Small</i>
HYB511	Cushion, <i>Small</i>	HYB523	Pillows, <i>Medium</i>
HYB513	Cushion, <i>Medium</i>	HYB525	Pillows, <i>Large</i>
HYB515	Cushion, <i>Large</i>	HYB530	Headgear

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Prodotto per RespCare Inc. Hybrid è un marchio registrato di RespCare, Inc. *IN ATTESA DI BREVETTAZIONE*