

AirFit™ F10

AirFit™ F10 For Her


Quattro™ Air

Quattro™ Air for Her

Mirage Quattro™

FULL FACE MASK FITTING TEMPLATE

Note: Mask fitting templates should be used to aide patients in the fitting of CPAP masks. Altering the size or dimensions of the original online PDF in any way may compromise the quality and accuracy of this template. The use of this template is not intended to replace the advice of a medical professional, sleep technician or HME dealer.

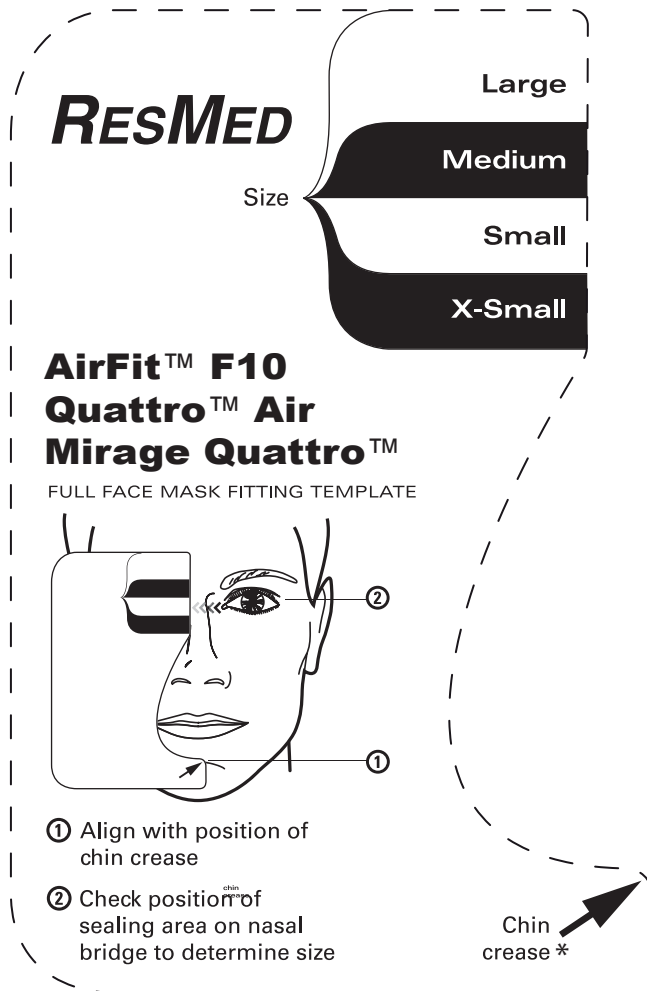


How to use the fitting template:

1. Cut out template along the dotted line
2. Follow the directions on the template to determine appropriate cushion size

* If chin crease falls between sizes, choose smaller size.



Presented With Compliments From
www.DirectHomeMedical.com

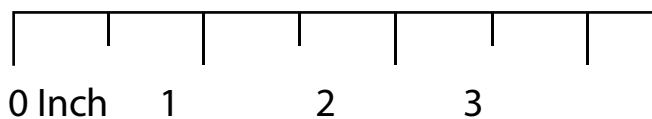

Actual Size: 3³/₈ inch (Width) X 5¹/₈ inch (Height)

Ruler is for scaling purposes. To ensure this print is accurate, match a ruler with the ruler above.