

# INOGENONE® G5

## External Battery Charger

### Guide



[CLICK TO LEARN MORE](#)



External Battery Charger Guide

# INOGENONE® G5 External Battery Charger

Your Inogen One® G5 External Battery Charger (BA-503) will charge your Inogen One® G5 single and double batteries. It takes approximately 4.5 hours to charge a fully depleted single battery, and approximately 9 hours to charge a fully depleted double battery. Note: the battery will also recharge in the Inogen One® G5 when powered by the AC Power Supply, or DC Power Cable.



1 Plug the External Battery Charger AC power supply cord into an electrical outlet.



2 Plug the External Battery Charger AC power supply into the battery charger.



3 Slide the charger onto the Inogen One® G5 Battery by clicking and locking into the charger.



4 When the battery is in the correct position, a solid red light will indicate that the battery is charging.



5 When the green light illuminates, the battery is fully charged\*.



6 Press battery latch down and slide charger off battery.

