



# CONCEPTS

## OXLIFE INDEPENDENCE

### QUICK REFERENCE GUIDE

THIS GUIDE IS INTENDED TO BE A QUICK REFERENCE ONLY. FOR COMPLETE USER INSTRUCTIONS PLEASE REFER TO THE USER MANUAL.

**DO NOT SMOKE WHEN USING THIS DEVICE. KEEP ALL MATCHES, LIT CIGARETTES, CANDLES OR OTHER SOURCES OF IGNITION AT LEAST 10 FEET FROM DEVICE.**

PLACE THE OXLIFE INDEPENDENCE IN A WELL VENTILATED AREA FREE OF SMOKE AND AWAY FROM DIRECT SUNLIGHT. ENSURE THAT AIR INTAKE AND EXHAUST PORTS ARE NOT OBSTRUCTED.


THE AVAILABILITY OF AN ALTERNATE SOURCE OF SUPPLEMENTAL OXYGEN IS STRONGLY RECOMMENDED.

### DEVICE SETTINGS

1 PRESS THE POWER BUTTON TO TURN DEVICE ON/OFF.

2 PRESS THE (M) BUTTON TO SELECT CONTINUOUS OR PULSE FLOW MODE.


3 PRESS THE INCREASE OR DECREASE BUTTONS TO SELECT THE CORRECT FLOW RATE AS PRESCRIBED BY YOUR DOCTOR.

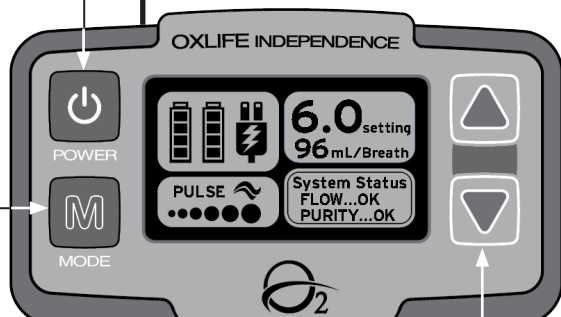

 DirectHome  
MEDICAL

Presented With Compliments From  
[www.DirectHomeMedical.com](http://www.DirectHomeMedical.com)

# BATTERY TIME MANAGEMENT

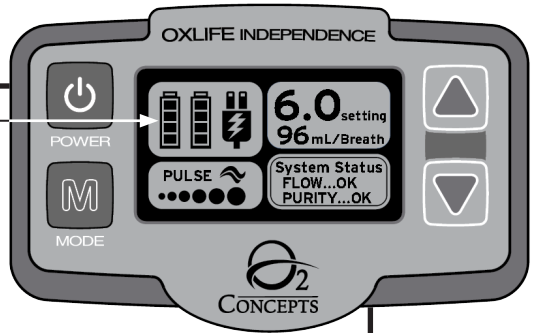


● **BATTERY CHARGE STATUS IS DISPLAYED IN THE UPPER LEFT PORTION OF THE CONTROL PANEL.**

- WITH THE COMBINED USE OF THE BATTERY, AC POWER CORD AND DC POWER CORD YOUR TIME AWAY FROM HOME IS VIRTUALLY LIMITLESS. USING THE AC AND DC POWER CORDS WHENEVER POSSIBLE WILL GIVE YOU OPTIMAL BATTERY LIFE.
- YOU WILL ACHIEVE LONGER BATTERY RUN TIME WHEN USING YOUR OXLife INDEPENDENCE IN PULSE FLOW MODE.
- PULSE FLOW MODE SHOULD ONLY BE USED UNDER THE DIRECTION OF YOUR PHYSICIAN.


## INSTALLING/REMOVING BATTERY

HOLD THE BATTERY AS SHOWN WITH THE CONNECTION TO THE BOTTOM, AND INSTALL IT INTO THE BACK OF THE DEVICE.

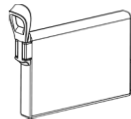


## TYPICAL BATTERY OPERATION TIMES (2 Batteries @ 20 Breaths/Minute)

FLOW RATE SETTING	CONTINUOUS FLOW		PULSE FLOW	
	1 BATTERY	2 BATTERIES	1 BATTERY	2 BATTERIES
0.5	2hr 53min	5hr 45min	3hr 8min	6hr 15min
1.0	2hr 38min	5hr 15min	3hr	6hr
1.5	1hr 45min	3hr 30min	3hr	6hr
2.0	1hr 15min	2hr 30min	2hr 53min	5hr 45min
2.5	1hr	2 hr	2hr 23min	4hr 45min
3.0	45min	1hr 30min	2hr	4hr
3.5	N/A		1hr 45min	3hr 30min
4.0			1hr 30min	3hr
4.5			1hr 15min	2hr 30min
5.0			1hr 8min	2hr 15min
5.5			1hr	2hr
6.0			53min	1hr 45min

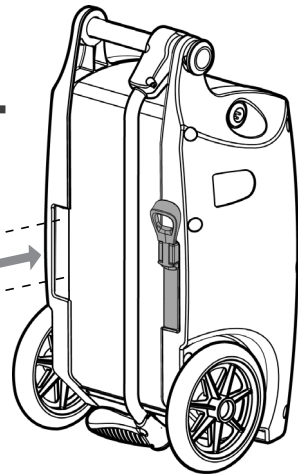


- **SLIDE INTO BACK OF DEVICE; BATTERY WILL DROP DOWN SLIGHTLY AND CLICK INTO POSITION.**

# EXTERNAL POWER

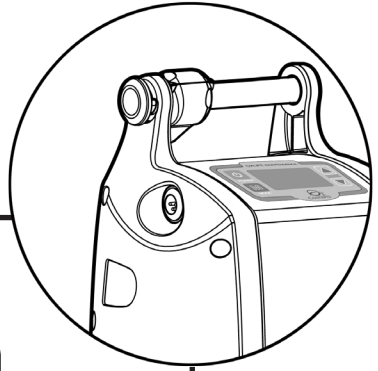

## AC POWER SUPPLY

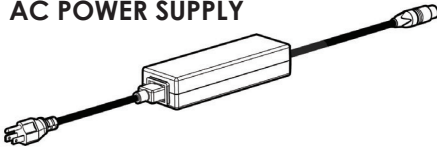

- TO USE THE AC POWER SUPPLY CONNECT THE POWER SUPPLY TO THE OXLIFE INDEPENDENCE AND WALL OUTLET. THE EXTERNAL POWER SOURCE ICON WILL BE DISPLAYED ON THE CONTROL PANEL.




## DC POWER SUPPLY




- THE DC POWER SUPPLY ALLOWS YOU TO POWER YOUR OXLIFE INDEPENDENCE FROM YOUR VEHICLE'S 12/15 VOLT DC OUTLET (CIGARETTE LIGHTER) IN ALL SETTINGS.
- **ALWAYS HAVE THE VEHICLE'S ENGINE RUNNING BEFORE PLUGGING IN YOUR OXLIFE INDEPENDENCE.**

- THE EXTERNAL POWER SOURCE ICON WILL BE DISPLAYED ON THE CONTROL PANEL.

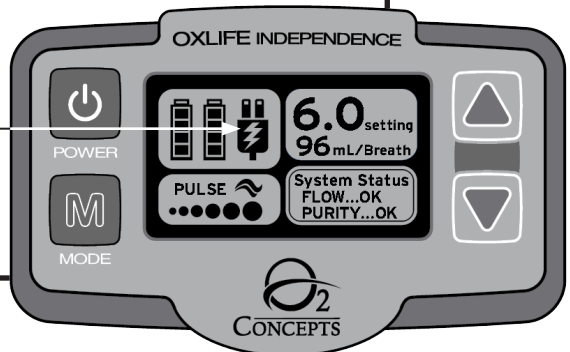

# HANDLE OPERATION

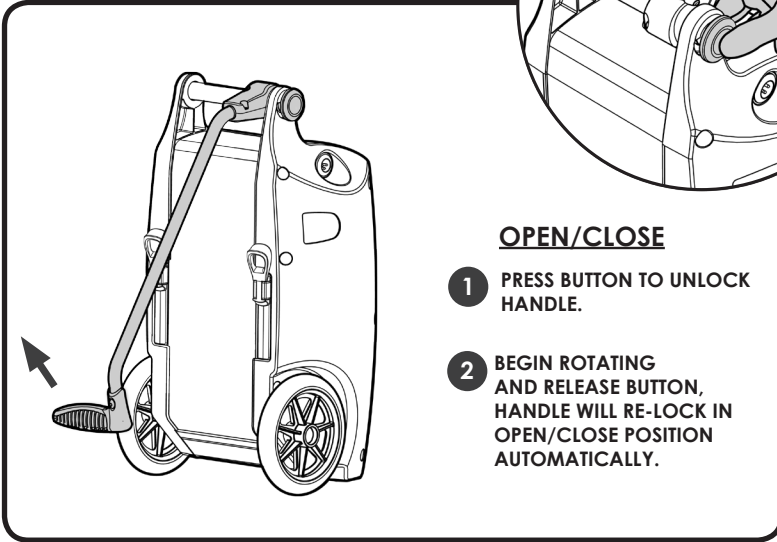

# FEATURE IDENTIFICATION

1. CONTROL PANEL
2. LCD DISPLAY
3. OXYGEN OUTLET PORT
4. EXHAUST PORT
5. WHEELS
6. RECHARGEABLE BATTERY SLOTS
7. FAA APPROVAL IDENTIFICATION
8. PULL HANDLE
9. AIR INTAKE PORT FILTER
10. EXTERNAL POWER INPUT
11. HANDLE LOCK/RELEASE BUTTON
12. CARRY HANDLE

