

Bubble Humidifier Technical Bulletin

7100 Bubble Humidifier- 3 PSI Pop-off | 7600 Bubble Humidifier- 6 PSI Pop-off

The Bubble Humidifier is a disposable, single-patient use device for use in health care facilities and home care. The humidifier is used to provide additional humidity to supplemental oxygen therapy and is available with either a 3 PSI or 6 PSI safety pop-off valve.

Product use

1. Wash hands and remove bubble humidifier from package.
2. Unscrew lid and add sterile water to the Maximum Level marking on the side of the bottle.
3. Replace lid and ensure lid is screwed on tight.
4. Attach bubble humidifier to oxygen source.
5. Test for leaks (A). Turn the flow to 6 LPM and block the outlet port on the bubble humidifier. An alarm should sound in 5 seconds or less. If not, check connections.
6. Attach oxygen tubing to the outlet port (B) and adjust flow to prescribed setting.
7. Apply interface, i.e. nasal cannula or oxygen mask, to patient.
8. If water level drops to Minimum level marking on bottle, add more sterile water.
9. Replace Bubble Humidifier every 14 days. Do not sterilize or reuse on more than one patient.



SLML-111 Rev B_Oct 2017

