




Presented With Compliments From
www.DirectHomeMedical.com

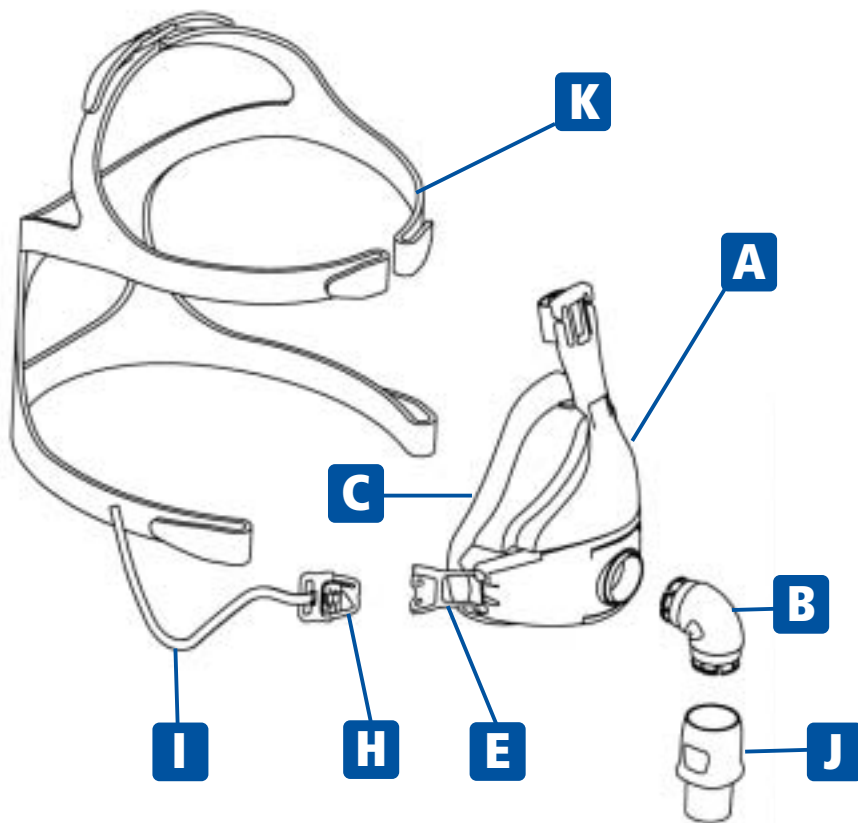

CLEANING YOUR MASK

To ensure continuing safety and comfort for your FlexiFit™ HC431 Mask:

Before each use:

- 1) Inspect the mask for deterioration. Do not use if aged or damaged.
- 2) Inspect the Non-Rebreathing Valve (J). Ensure the silicone flaps occlude the pipe when the CPAP is off.

After each use:

- 1) Disassemble the Silicone Seal (C), Elbow (B), Non-Rebreathing Valve (J), and the Headgear (K) from the Mask Base (A).
- 2) Wash the Silicone Seal, Elbow, Non-Rebreathing Valve and Mask Base in warm soapy water then rinse in fresh water.
- 3) Leave all parts to dry, away from direct sun light before reassembling.

Every 7 days:

- 1) Hand wash the Headgear in lukewarm water using a mild detergent. Allow to air dry out of direct sun light. It is not necessary to remove the Clip (H) from the headgear when washing.

Caution:

Do not clean the mask in the dishwasher.

Do not clean the mask with alcohol, chlorine or bleach.

Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ HC431 weakens or cracks, seek replacement immediately.

ASSEMBLING YOUR MASK

Your FlexiFit™ HC431 Mask is fully assembled and ready to use. However, when you need to clean your mask follow these instructions for reassembly.

- 1) Place Silicone Seal (C) into Mask Base (A)
- 2) Push the Non-Rebreathing Valve (J) onto the Elbow (B), and attach to Mask Base.
- 3) Lay the Headgear (K) flat with the label side down. Position the assembled mask over the Headgear and attach the four straps in to the corresponding slots in the Mask Base and the Glider™ strap (E). This can be done without undoing the Velcro® Tabs by sliding the Headgear into the slots.